User Manual

DEALER: This manual MUST be given to the user of the product.

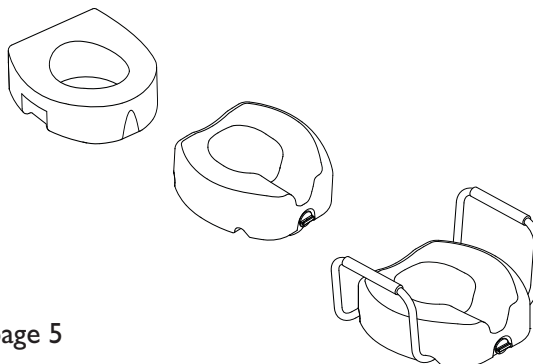
USER: BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Raised Toilet Seat

MODEL	DESCRIPTION
I300RTS	Raised Toilet Seat
I301RTS	Raised Toilet Seat with Adjustment Knob
I302RTS	Raised Toilet Seat with Adjustment Knob and Arms

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I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

2 Safety

Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/ dealer for further instruction.

2.1 Weight Limitation

WARNING

ALWAYS observe the weight limit on the labeling of your Raised Toilet Seat. Check that all labels are present and legible. Replace if necessary.

The weight limitation of the I300RTS, I301RTS and I302 RTS Raised Toilet Seat is 300 pounds (137 kg).

Part No I167433

I

Raised Toilet Seat

2.2 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

ALWAYS make sure that the Raised Toilet Seat is securely in place and stable before each use. Although the Raised Toilet Seat is designed for use on most popular toilets, it may not work on certain unusual or artistic bowls.

Be sure that the user's weight is centered on the Raised Toilet Seat as the seat may tip if the user leans too far forward or from side to side.

Invacare recommends the use of a toilet safety frame or grab bars in addition to the Raised Toilet Seat.

3 Removing/Installing the Raised Toilet Seat

For this section, refer to FIGURE 1 on page 3.

To remove the Raised Toilet Seat, reverse this procedure.

The existing toilet seat and lid can be flipped up or removed depending upon the amount of usage.

Perform one of the following procedures depending on the Raised Toilet Seat type:

Model I300RTS - Raised Toilet Seat

1. Put the Raised Toilet Seat in place on the toilet bowl with the tall end of the seat towards the back of the toilet bowl. Refer to DETAIL "A".
2. Properly seat the inner ring inside the toilet bowl.
3. Check the stability of the Raised Toilet Seat. Realign and/or adjust as necessary.

Model I301RTS - Raised Toilet Seat with Adjustment Knob

1. Put the rear wings of the Raised Toilet Seat in place underneath the 1-3/4 inch toilet lip at the back of the toilet bowl.
2. Rotate the Raised Toilet Seat down until the front of the seat rests on the toilet bowl.
3. If necessary, rotate the adjustment knob counterclockwise until the bracket fits inside the toilet bowl.
4. Turn the adjustment knob clockwise to tighten the bracket against the inner surface of the toilet bowl.
5. Check the stability of the Raised Toilet Seat. Realign and/or adjust as necessary.

Model I302RTS - Raised Toilet Seat with Adjustment Knob and Arms

1. Remove the plastic from the arms.
2. Install the arms onto the Raised Toilet Seat.
 - A. Depress both snap buttons on the arm and insert the arm into the mounting tubes.
 - B. Slide the snap buttons to the appropriate holes in the mounting tubes.
- i** There will be an audible “click”. Ensure the snap buttons fully protrude through the holes in the mounting tubes.
3. Repeat STEP 2 to install the opposite arm onto the Raised Toilet Seat.
4. Put the rear wings of the Raised Toilet Seat in place underneath the 1-3/4 inch toilet lip at the back of the toilet bowl.
5. Rotate the Raised Toilet Seat down until the front of the seat rests on the toilet bowl. If necessary, rotate the adjustment knob counterclockwise until the bracket fits inside the toilet bowl.
6. Turn the adjustment knob clockwise to tighten the bracket against the inner surface of the toilet bowl.

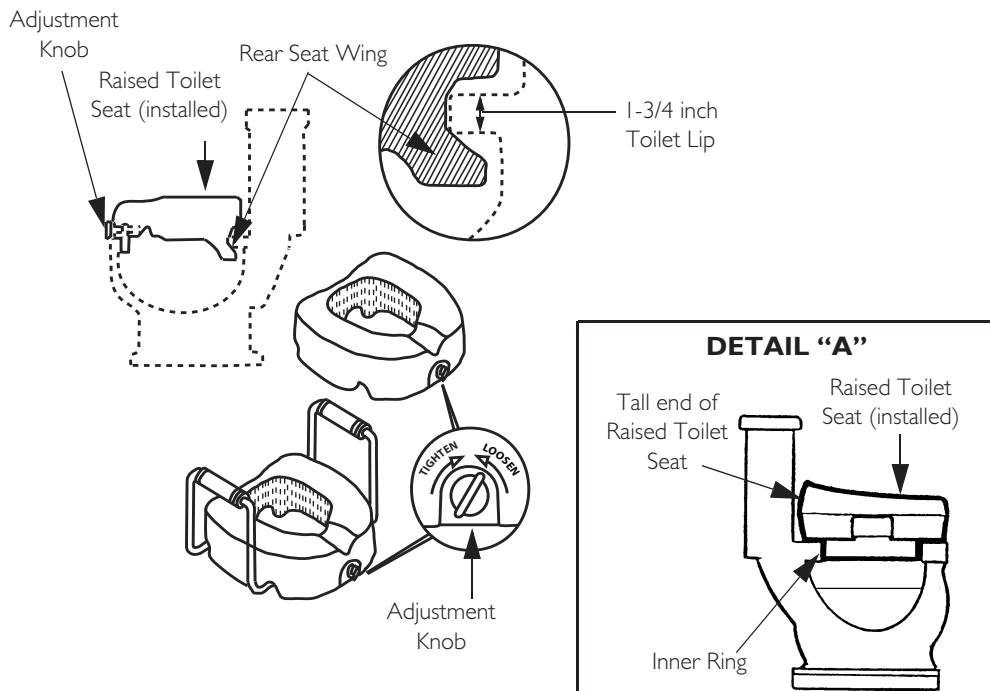
FIGURE I Removing/Installing the Raised Toilet Seat

4 Care and Maintenance

- i** DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the Raised Toilet Seat.
1. Regular cleaning of the Raised Toilet Seat is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
2. Regular inspection of parts for deformation, corrosion, breakage, wear or compression is recommended. Replace the Raised Toilet Seat if any of these conditions exist.

Part No I167433

3

Raised Toilet Seat

5 Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state. Invacare warrants all this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2010 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.