

Heavy Duty Cleaning Procedure



CONTINUING CARE GROUP

Maintenance Instructions

For Invacare Continuing Care Group Tubs

SAVE THESE INSTRUCTIONS

WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

The disinfectant dispensing system has been factory tested and calibrated for use with Invacare disinfectant. The use of any disinfectant with a dilution ratio (½ oz. per gallon) or viscosity level different than that of Invacare disinfectant will result in an improper mixture of disinfectant and water. This may cause your system to be improperly disinfected - or in some instances, not disinfected at all. If using any disinfectant with a dilution ratio or viscosity level different than that of Invacare disinfectant or if switching from one type of disinfectant to another type, the disinfectant flowmeter must be calibrated before performing this procedure. Refer to the tub owner's manual or to the calibration instruction sheet, part number 1154272, for the calibration procedure.

ALWAYS wear rubber gloves, an apron and face shield protection during disinfecting, cleaning and rinsing procedures. In case of eye or skin contact, follow procedures on disinfectant container.

CAUTION

NEVER use abrasive cleaners like scouring powder or liquid cleaners containing pumice stone. The use of these types of cleaners will make the gel-coat finish of your tub scratched and dull. **NEVER** use cleaners containing iodine, bromine, Betadine® or methylene blue. The use of these types of cleaners will cause the tub to stain. Never use cleaners containing bleach. Use of these types of cleaners will dry out the rubber seals and gaskets and the tub will not function properly.

Before starting the whirlpool pump, the water level must be completely above the whirlpool jets. Otherwise damage to the motor/pump will result.

Heavy Duty Cleaning Procedure

In addition to disinfecting your tub after every use, you will need to perform this procedure approximately once every 80-150 baths.

1. Clean the entire surface of the tub to remove visible tissue residue and fluid.
2. Rinse the inside surface of the tub with the shower/massage wand on the spray setting.
3. Rotate the water temperature valve to adjust the water temperature to approximately 105° F.

NOTE: This will maximize the cleaning solution's ability to clean and disinfect the system.

4. Close the drain.
5. Make sure the whirlpool jets are open.
6. Fill the tub one inch above the highest whirlpool jets.
7. Add the Invacare Heavy Duty Cleaner to the water in the tub. Refer to the chart for the proper amount.
8. Spray pre-mixed Invacare Heavy Duty Cleaner on the remaining surfaces.
9. Press the whirlpool on/off button to turn on the whirlpool pump and let the pump run for 60 seconds.
10. Turn off the pump and allow the heavy duty cleaner/water solution to remain in the tub for 10 minutes.
11. During this ten-minute waiting time, take a long handled brush and thoroughly scrub all interior surfaces of the tub.
12. Open the drain to allow the heavy duty/water solution to drain from the tub.
13. Use the shower/massage wand on the pulse setting and direct the spray into both whirlpool jets to back flush until the water discharging from the whirlpool inlet is clear.
14. Rinse the tub thoroughly with the shower wand on the spray setting.
15. Check the tub for damage and to ensure all surfaces of the tub are clean and that no visible residue remains. If residue is found, repeat STEPS 13 and 14.

TUB MODEL NUMBER	WATER REQUIRED TO COVER JETS (GALLONS)	INVACARE HEAVY DUTY CLEANER REQUIRED (OZ)*
2800	10	40
2001	32	128
3700	30	120
3750	30	120
3800	30	120
3600	30	120
3650	48	192
3600XL	48	192
2300	25	100
3300	30	120
4300	35	140
4300HL	35	140
5300	52	208
5300HL	52	208
6300	50	200
6300ADH	50	200
4400SS	48	192
5400SS	50	200

**NOTE: If using non-Invacare heavy duty cleaner, refer to the instructions supplied with the cleaner to determine the proper amount required.*

Invacare Continuing Care Group

www.invacare-ccg.com



CONTINUING CARE GROUP

USA
One Invacare Way
Elyria, Ohio USA
44036-2125

Canada
994 Hargrieve Rd.
London Ontario
N6E 1P5 Canada

All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.

Betadine is a registered trademark of Purdue Pharma L.P.
© 2008 Invacare Corporation

Part No 1154271

Rev A - 04/08

For customer service call: 800-668-2337

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>