# Hydraulic/Electric Patient Lift Warning Label Kit

## Assembly, Installation and Operating Instructions

Kit No. I130270

Yes, you can!

### SAVE THESE INSTRUCTIONS

## Safety Summary

To ensure the safe and proper installation of the WARNING/CAUTION labels, these instructions MUST be followed:

### **⚠ WARNING**

**DO NOT** install or use this kit without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may occur.

**DO NOT** use the patient lift without installing these Warning and Caution labels.

The label kit contains the following labels:

PART NUMBER	DESCRIPTION OF LABEL	REPLACE LABEL
I130268B	Patient Lift Warning Label 450	I073698B
I130230B	Patient Lift Warning Label 600	I070965A
I130266A	Mast Bolt Warning Label	I130200A

## Installing Warning/Caution Labels

1. Determine the proper location for each label. Refer to FIGURE 1.

*NOTE: The English labels are included in FIGURE 1 for ease in determining proper location.*

2. Peel one (1) label from the adhesive backing.
3. Position the appropriate warning or caution label directly over the English label at the location determined in STEP 1.
4. Repeat STEPS 1-3 for each label.

For more information regarding Invacare products, parts, and services,  
please visit [www.invacare.com](http://www.invacare.com)

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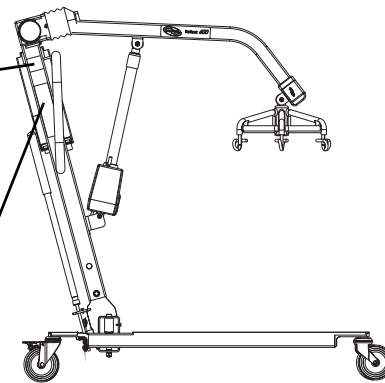
**⚠ WARNING**  
Mast pivot under rubber boot must be tight to ensure safe use of your patient lift. Bolt must be checked at least every six (6) months in conjunction with periodic maintenance. See Owner's Manual.  
P/N 1130200 Rev. A 01/05

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*NOTE: Electric Lift shown. Locations are the same on the Hydraulic/Manual lift.*

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*NOTE: For 600-lb Lifts*  
*NOTA: Para elevadores de 272 kg (600 lb)*

*NOTE: For 450-lb Lifts*  
*NOTA: Para elevadores de 204 kg (450 lb)*

**⚠ WARNING**  
  
**BEFORE** using the Patient Lift, **READ** and **UNDERSTAND** the Owner's Manual for proper operation and safety procedures.  
**⚠ WARNING**  
**WEIGHT LIMITATION 600 lbs.** The Invacare Patient Lift is **NOT** a transport device. Moving a person suspended in a sling over **ANY** distance is **NOT** RECOMMENDED.  
**DO NOT** roll casterbase over deep carpet, raised carpet bindings, door frames or any uneven surfaces that may cause the Patient Lift to tip over.  
**DO NOT** lock the casters of the Patient Lift when lifting an individual. Casters **MUST** be left unlocked to allow Patient Lift to stabilize during lifting procedures.  
**USE ONLY** genuine INVACARE SLINGS and LIFT ACCESSORIES.  
**USE ONLY** RELIANT 600 Model Components for maintenance and replacement.  
Refer to Owner's Manual for periodic maintenance procedures.  
Customer Service:  
1-800-333-6900  
Rev. A 04/05 1070965

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**⚠ WARNING**  
  
**BEFORE** using the Patient Lift, **READ** and **UNDERSTAND** the Owner's Manual for proper operation and safety procedures.  
**⚠ WARNING**  
**WEIGHT LIMITATION 450 lbs.** The Invacare Patient Lift is **NOT** a transport device. Moving a person suspended in a sling over **ANY** distance is **NOT** RECOMMENDED.  
**DO NOT** roll casterbase over deep carpet, raised carpet bindings, door frames or any uneven surfaces that may cause the Patient Lift to tip over.  
**DO NOT** lock the casters of the Patient Lift when lifting an individual. Casters **MUST** be left unlocked to allow Patient Lift to stabilize during lifting procedures.  
**USE ONLY** genuine INVACARE SLINGS and LIFT ACCESSORIES.  
**USE ONLY** RELIANT 450 MODEL Components for maintenance and replacement.  
Refer to Owner's Manual for periodic maintenance procedures.  
Customer Service:  
1-800-333-6900  
1073698 REV. B 07/05

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© 2005 Invacare Corporation
trademarks of Invacare Corporation.
**Invacare Corporation** [www.invacare.com](http://www.invacare.com)