WALKLITE™ WALKER (MODEL 6300-5F)
MARCHETTE WALKLITE (MODÉLE 6300-5F)
ANDADOR WALKLITE (MODELO 6300-5F)

Refer to page 2.
Se référer à la page 11.
Refiérase a la página 22.

Walklite Walkers provide assistance for individuals weighing up to 300 lbs (136 kg).

Walklite Walker

Model No.	Width		Depth		Height Adjustment	Approximate Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
6300-5F	18" (46cm)	20-1/2" (52cm)	23" (58cm)	7" (18cm)	32" - 36" (81cm-91cm)	5'5" - 6'0" (165cm-178cm)

Specifications are subject to change without notification.

**For more information regarding
Invacare products, parts, and services,
please visit: www.invacare.com**

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the safe operation of the Walklite Walker, these instructions MUST be followed:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

GENERAL WARNINGS

If walker is exposed to extreme temperature (about 100°F [38°C] or below 32°F[0°C]), high humidity and/or becomes wet, prior to use, ensure handgrips do not twist on side frame otherwise damage or injury may occur.

Walklite Walkers are **NOT** intended to be self propelled while seated.

A physical/occupational therapist should assist in the height adjustment of the walker for maximum support.

Care should be taken to ensure that **ALL** hand and height adjustments are secure, and that wheels and moving parts are in good working order before using this or any mobility aid.

All wheels and glide brakes **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the walker is properly balanced.

The Rear Glide Brakes are intended to provide additional security when in the seated position. The rear glide brakes are **NOT** intended to be used for stopping the walker during ambulation.

For optimal stability, the seat should be in the **DOWN** position when using the walker for ambulatory assistance.

Always use caution when using walker on wet or icy surfaces.

STABILITY WARNINGS

The Walklite Walker can provide ambulatory assistance to an individual weighing up to 300 lbs (136 kg), **INCLUDING** the weight of the contents of the basket, if so equipped.

STABILITY WARNINGS

The Walklite Walker basket has a weight limitation of 10 lbs (4.5 kg).

The Rear Glide Brakes engagement may vary depending upon the weight distributed on the seat and the positioning of the user on the seat.

DO NOT rock the walker while sitting on the walker seat.

DO NOT lean back or tip the walker while sitting on the walker seat.

FOLDING/ASSEMBLY WARNINGS

After unfolding or assembling Walklite Walker, ensure that walker is securely locked in **OPEN** position and level to the ground **BEFORE** using.

DO NOT hang anything on either the left or right side frames (area that includes dual release paddles). This will cause side frames to bend downward and prevent walker from locking into place when fully opened. This may also cause walker to tip, resulting in injury or damage.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

Always test to see that the walker and attachments are properly and securely locked in place **BEFORE** using.

OPENING THE WALKER (FIGURE 1)

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. To OPEN the walker, pivot the left and right side frames outward until the release paddles activate (engage upward).
2. Grasp the handle and lower the seat, pushing DOWN until the seat supports fully engage the side braces as shown in FIGURE 1.
3. Before using, test walker by grasping the side frames and, with some force, attempt to fold them inward. The walker will remain locked in the OPEN position if it is engaged properly.


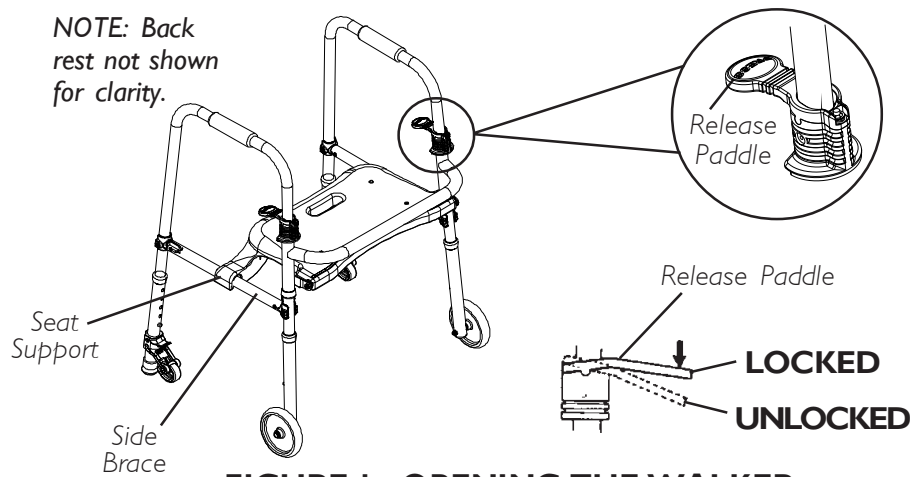
FIGURE 1 - OPENING THE WALKER

FOLDING THE WALKER (FIGURE 2)

1. Raise the seat by grasping the handle and lifting UP.
2. Depress one (1) release paddle and pivot the corresponding side frame inward toward the front crossbrace.
3. Repeat the same procedure for the remaining side frame.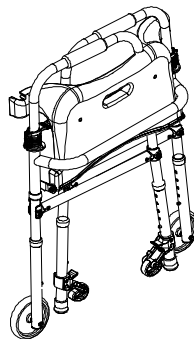

NOTE: Back rest not shown for clarity.

FIGURE 2 - FOLDING THE WALKER

INSTALLING/ADJUSTING LEG EXTENSIONS (FIGURE 3)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: For the most comfortable position, make sure the adjustment is performed while wearing the most frequently worn shoes.

NOTE: Adjust the walker height so that when the user's arm is down to their side, the hand grip is at wrist height. This will ensure the arms are at an approximate 20° - 30° bend when using the walker.

-
1. Depress the snap buttons on the front leg extensions (with wheels) of the walker and slide the leg extensions up or down to the desired height. Ensure that the snap button protrudes through the appropriate adjustment hole on the leg extension (FIGURE 3).

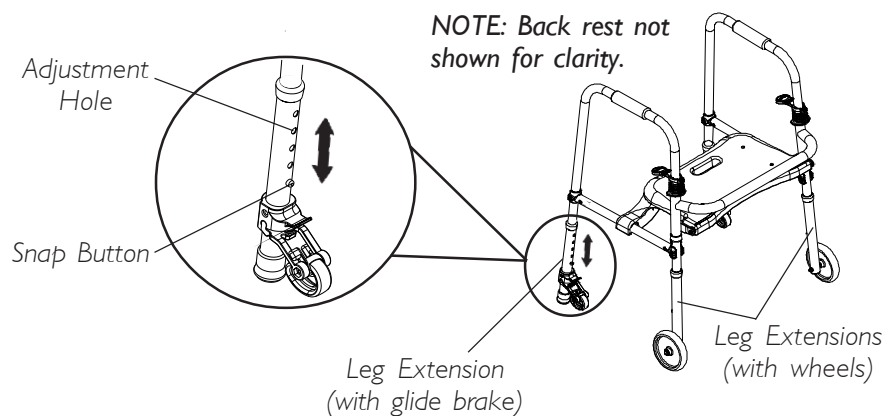
FIGURE 3 - INSTALLING/ADJUSTING LEG EXTENSIONS

2. Repeat STEP 1 for the rear leg extensions (with glide brakes).
3. Ensure all four (4) legs are adjusted so that the walker is level.

WARNING

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch (2.5cm) lower than the front.

USING GLIDE BRAKES (FIGURE 4)

NOTE: Refer to all WARNINGS in the SAFETY SUMMARY in this instruction sheet.

NOTE: Only the glide brake heels should touch the ground/floor when walking with walker.

TO ENGAGE THE GLIDE BRAKES

1. Stop moving the walker.
2. Ensure that the seat is properly placed DOWN. Refer to OPENING THE WALKER in this manual.
3. Carefully sit on the seat. Adjust your weight so it is evenly distributed on the walker seat.

NOTE: The user's weight on the seat causes the glide brakes to engage.

WARNING

The Rear Glide Brakes engagement may vary depending upon the weight distributed on the seat and the positioning of the user on the seat.

DO NOT rock, lean back, or tip the walker while seated on the walker seat.

TO RELEASE GLIDE BRAKES

1. Grab both side handles and carefully lift yourself up into a standing position while shifting your weight FORWARD.

NOTE: When the user's weight is lifted off of the seat, the glide brakes disengage.

WARNING

DO NOT lean backward while going from a seated to a standing position otherwise the walker may move.

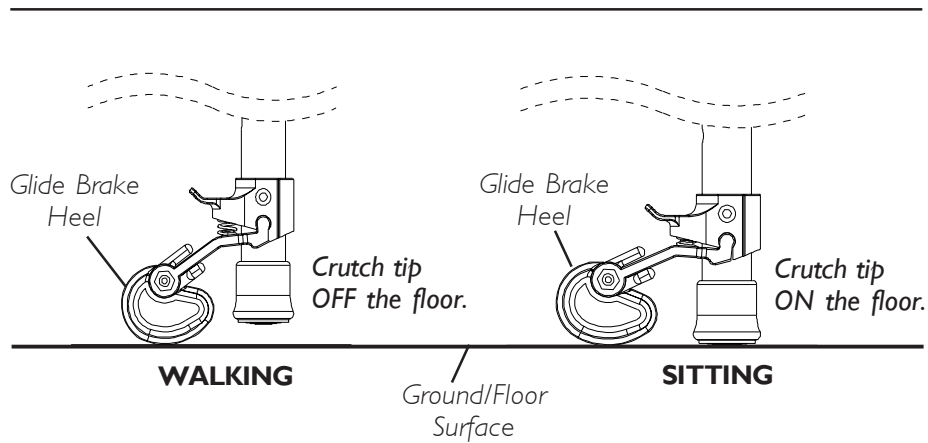
FIGURE 4 - USING GLIDE BRAKES

CARE AND MAINTENANCE

1. Ensure that **ALL** attaching hardware is secure at **ALL** times.
2. Check screw securing seat to walker for tightness. Tighten if necessary.
3. Make sure snap buttons release and re-engage properly.
4. Replace any broken, damaged or worn items immediately.
5. Check hand grips to ensure they do not twist on side frame. If hand grips move, contact your Invacare dealer for service.

CARE OF WHEELED ACCESSORIES

1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
2. Lubricate with 3-in-1[®] oil periodically or if wheels begin to squeak.

LIFETIME LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our product.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the frame to be free from defects in materials and workmanship for the lifetime of the product, and one (1) year for all other remaining components, excluding tires/wheels and normal wear of the glide brakes, for the original purchaser/user. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, IMPROPER MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER. SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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REGISTER YOUR PRODUCT!

The benefits of registering:

1. Safeguard your investment.
2. Ensure long term maintenance and servicing of your purchase.
3. Receive updates with product information, maintenance tips, and industry news.
4. Invacare can contact you or your provider, if servicing is needed on your product.
5. It will enable Invacare to improve product designs based on your input and needs.

Register ONLINE at www.invacare.com

- OR -

Detach form, complete and mail.

Any registration information you submit will be used by Invacare Corporation only, and protected as required by applicable laws and regulations.

Name _____

Address _____

City _____ State/Province _____

Zip/Postal Code _____

Email _____ Phone No. _____

Invacare Model No. _____

Serial No. _____

Purchased From _____

Date of Purchase: _____

1. Method of purchase: (check all that apply)
 Medicare Insurance Medicaid Other
2. This product was purchased for use by: (check one)
 Self Parent Spouse Other
3. Product was purchased for use at:
 Home Facility Other
4. I purchased an Invacare product because:
 Price Features (list features)
5. Who referred you to Invacare products? (check all that apply)
 Doctor Therapist Friend Relative Other
 No referral Advertisement (circle one): TV, Radio, Magazine, Newspaper
6. What additional features, if any, would you like to see on this product?

7. Would you like information sent to you about Invacare products that may be available for a particular medical condition? Yes No

If yes, please list any condition(s) here and we will send you information by email and/or mail about any available Invacare products that may help treat, care for or manage such condition(s):

8. Would you like to receive updated information via email or regular mail about the Invacare home medical products sold by Invacare's dealers? Yes No


9. What would you like to see on the Invacare website?

10. Would you like to be part of future online surveys for Invacare products?
 Yes No
11. User's Year of birth: _____

If at any time you wish not to receive future mailings from us, please contact us at Invacare Corporation, CRM Department, 39400 Taylor Parkway, Elyria, OH 44035, or fax to 877-619-7996 and we will remove you from our mailing list.

**To find more information about our products, visit
www.invacare.com.**

Please Seal with
Tape Before Mailing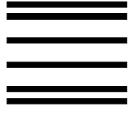


BUSINESS REPLY MAIL

FIRST-CLASS MAIL PERMIT NO. 196 ELYRIA OH

POSTAGE WILL BE PAID BY ADDRESSEE

INVACARE CORPORATION
CRM DEPARTMENT
39400 TAYLOR PARKWAY
ELYRIA OH 44035-9836

NO POSTAGE
NECESSARY
IF MAILED
IN THE
UNITED STATES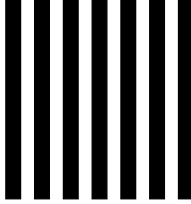

Invacare Product Registration Form

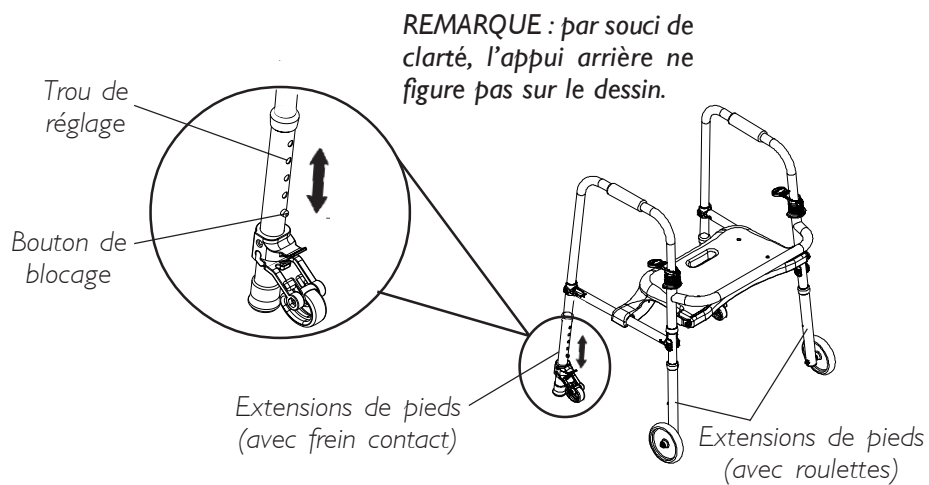
FIGURE 3 - INSTALLATION/RÉGLAGE DES EXTENSIONS DE PIEDS

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© 2004 Invacare Corporation
Invacare is a registered trademark of
3-in-1 is a registered trademark of
Yes, you can and Walklite are trademarks
American Home Products Corporation.
of Invacare Corporation.