*Instructions  
for use*

*Instructivo  
de uso*

**Double Deluxe™**

**Breastpump • Bomba extractora**

# IMPORTANT SAFEGUARDS

***This is a single user product. Use by more than one person may present a health risk, performance questions and voids the warranty.***

***When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:***

## READ ALL INSTRUCTIONS

**DANGER:** To reduce the risk of electrocution:

1. Always unplug electrical products immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product if it has fallen into water. Unplug immediately.

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged into an electric outlet.
2. Close supervision is necessary when this product is used by, on or near children or invalids.
3. Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Keep the cord away from heated surfaces.
6. Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
7. Plug the transformer into Double Deluxe first and then into wall socket.
8. Only use the transformer that comes with the Double Deluxe Breastpump.
9. This product is a personal care item and shouldn't be shared between mothers.
10. Make sure the voltage of the transformer is compatible with the power source.
11. Never use while sleeping or drowsy.
12. Do not use the Vehicle Lighter Adapter with a Double Deluxe Breastpump.

## SAVE THESE INSTRUCTIONS

### English

#### Table of Contents

Uses	1
Health & Safety Information	2
Cleaning, Hygiene, Storing Breastmilk	
Assembly Instructions	4
Battery & Electric Operation	
To Begin Pumping	
Pumping Tips	6
Troubleshooting	7
Accessories	7
Warranty	8
Ordering Parts	9
Breastpump Parts List	9
Parts	21

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***Disassemble all parts prior to cleaning and sterilization. Especially be sure to remove the valve from the breastshield frame and separate the white membrane from the valve prior to cleaning or sterilization. See parts 5 and 6 on breastpump parts lists (Page 9).***

## Uses

- Collect breastmilk during short-term occasional absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement or sore nipples. Pumping with simultaneous massage is helpful to relieve engorgement.

## IMPORTANT

***This product has been designed for single user use and should not be shared between mothers.***

**NOTE:** For **long-term or frequent** pumping; when the baby is not available to feed at the breast at least part of the time every day (a premature baby, for example), use of a full-size electric pump (Lactina® or Classic™ Breastpump) is suggested. These pumps can be rented inexpensively for home use. For the names of rental stations or breastfeeding specialists in your area, call **1-800-TELL YOU** (1-800-835-5968).

**IMPORTANT:** When using the pump, do not overfill or tilt the containers.

## Health & Safety Information

Please read this section before using any Medela Breastpump.

### Cleaning & Sanitizing

Prior to first use and before use each day, follow these cleaning instructions at home unless you are otherwise instructed by your physician or another medical professional from your hospital.

#### Boil

Disassemble, wash, and rinse all parts (except motor unit) that come in contact with the breast or the milk. Boil cylinder/piston assembly, including rubber parts, weekly or when milk spots appear.

- Place parts in a 4-5 quart pan.
- Fill to within 1 inch of rim with cold water. The use of distilled water will help to minimize deposits onto components.
- Cover and boil for 20 minutes.
- Remove from stove. Drain water off. Allow parts to cool in open pan.
- Remove all parts and allow to air-dry.

**Use of microwave sterilization is not recommended and may damage some components.**

After each use:

#### Wash

Disassemble and wash all parts that come in contact with the breast and milk in soapy water or wash in top-rack of dishwasher. The motor unit may be wiped with a damp cloth.

#### Rinse

Rinse in clear water.

#### Air-dry

Air-dry on clean towel and cover parts when not in use. The tubing should be

hung to air-dry. For faster drying, attach tubing to the pump and run for 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubing to dry. Carefully swinging the tubing in a circular motion, will also help to speed drying.

### Tubing Care

Inspect tubing after each pumping session for condensation and/or milk.

#### If condensation appears in tubing:

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry.

### Hygiene

Wash hands thoroughly before pumping. If your physician recommends, clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.

### Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

### In a refrigerator:

- To inhibit the growth of bacteria, breastmilk which is not used immediately should be refrigerated. Milk can be refrigerated for up to 5-7 days or, if longer storage is needed, freeze the milk.

### In a refrigerator freezer compartment:

- In the freezer compartment inside a refrigerator, breastmilk can be stored for approximately six months.

### In a separate freezer:

- In a separate -20° C freezer, breastmilk can be stored for approximately 12 months.

### Thawing frozen breastmilk:

- Thaw frozen breastmilk by holding the container under warm water. Milk may be defrosted in the refrigerator overnight. Gently shake the container to blend any fat that has separated.

### Caution:

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- If adding expressed breastmilk to container of already frozen breastmilk, make sure to add a less amount than the already frozen amount.

## Assembly Instructions

1 Snap the white membrane onto the valve until membrane lies completely flat.

**Be sure to separate membranes and valves when cleaning and completely dry prior to reassembly.**

2 Press the valve assembly onto the breastshield.

3 Screw the container onto the breastshield or use disposable CSF™ Collection, Storage & Freezer Bags article #87010 or #87013 (sold separately).

4 Align the raised line on the motor unit with the raised line on the shield and press the motor firmly onto the shield with the single adapter attached.

5 Lock the motor onto the shield by rotating 1/8 turn clockwise. The white plug-in port on the motor unit should be aligned with the hole on the breastshield.

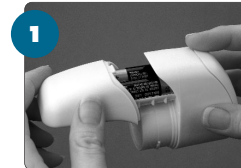
6 For **single pumping**, insert the small plug of the single adapter into the hole on the breastshield and firmly push into the plug-in port on the motor unit. This plug should be inserted into the holder when not in use.

7 For **double pumping**, insert the smaller end of the tubing into the plug-in on the motor unit, and the larger end of the tubing into the second breastshield (the one not attached to the motor unit)

## Battery Operation

1 To install batteries, slide off the battery compartment cover. Install two fresh AA 1.5 V batteries (alkaline preferred).

While the pump will work with either alkaline, disposable or nicad rechargeable batteries, alkalines are recommended; the pump will cycle more quickly, pump longer and run more quietly.

### CAUTION:

When using the pump with batteries, the pumping cycle may slow as the batteries wear down. Slower cycles may cause some discomfort because the vacuum will be held longer on the breast. If this happens, reduce the vacuum with the adjustment regulator and replace the batteries with fresh ones as soon as possible.

### NOTE:

The batteries do not need to be removed to use electricity to power the pump.

## Electric Operation

Insert the metal plug on the end of the transformer cord into the hole on the side of the motor unit.

(Use only Medela transformer #9200013).

Center the shield(s) over your nipple(s). Start the pump using the “ON/OFF” switch on the back of the motor.

**NOTE:** Be sure to push switch all the way to the top to assure adequate suction.

Begin pumping with the vacuum regulator set on “MIN” and gradually increase to your comfort level.

## Pumping Tips

It is important that you be as comfortable and relaxed as possible when pumping. Learning relaxation techniques may be helpful. Many mothers find that focusing on a photograph of their baby is useful.

When double pumping, it may take about 10-15 minutes to collect milk from both breasts. Single pumping may take 10-15 minutes per side.

### ***The following steps of caution should be followed to prevent milk overflowing into the motor:***

- Care should be taken not to over fill the container(s).
- Do not tilt or lay the container down when milk is in it and the tubing is attached.
- Remove the tubing from the breastshield before laying the breastshield down.

## Troubleshooting

***If milk accidentally gets into the tubing/motor, immediately follow these steps:***

- Remove motor from the breastshield.
- Wipe milk off the motor with a paper towel or tissue.
- Run motor for at least 30 seconds to clear out any milk that remains inside.
- Repeat step 2. Take special care to clean the vent holes in the front and side of the motor.
- Clean the breastshield where it attaches to the motor.
- Put the motor back on the breastshield and resume pumping. When pumping is completed, repeat these steps.

### ***If There Is No Vacuum Power***

If you do not feel any vacuum at the breast or if the pumping action is too weak:

- Check that the breastshield forms a complete seal around the breast.
- Check to see that the white membrane is securely seated into the valve head on the underside of the breastshield frame. Ensure that membrane is clean and that the flap is moveable and not dried onto the valve head.
- For single pumping, check to see that the single adapter plug is firmly secured into the plug-in port on the breastshield/ motor unit.
- For double pumping, Be sure that the tubing ends are firmly seated into their appropriate plug-in points.
- Check that the gaskets are clean and dry.
- Check with fresh batteries or electrical if not powering unit with batteries.
- Check that the connector tube is firmly plugged into the breastshield.

## Accessories

### **CSF™ Bags**

#87010, #87013 (US & Canada), #47013, 47010 (Latin America)

Collect, store and freeze breastmilk in these sterilized bags. Pump directly into bags assuring maximum hygiene for baby. (Breastshield sold separately)

## 90-Day Limited Warranty

This product is warranted by Medela, Inc., to the original retail purchaser to be free from defects in material and workmanship for 90 days from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

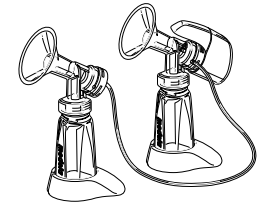
THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

You may call 1-800-435-8316 for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela, prepaid, together with your dated bill of sale or other proof of purchase, a brief statement of the problem and your return authorization number to the following address:

**Attn: Medela, Inc.  
P.O. Box 660  
McHenry, IL 60051-0660 USA**

## Ordering Parts

Many Medela locations have parts available for sale. Call the location where you purchased the breastpump or call 1-800-TELL YOU in USA or Canada for a location in your area. Or call 1-800-435-8316 or 1-815-363-1166 extension 114 and order with VISA, MasterCard, American Express, or Discover Card. Prices subject to change without notice.

## Breastpump Parts List (Page 21)

Item	Order Number
1. Motor Unit .....	#0062010
2. Transformer .....	#9200013
3. Breastshield .....	#8100373
4. Single Adapter .....	#8100471
5. Valve Head .....	#8100475
6. Membrane .....	#8207013
7. 5oz. (150ml) Container .....	#8117009
8. Container Stand .....	#8100462
9. Container Lid .....	#8107057
10. Battery Cover .....	#8100464
11. Tubing Assembly .....	#8007102

\* Prices subject to change without notice. Prices and availability vary outside USA.

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Medela and Lactina are registered trademarks of Medela. Classic and CSF are trademarks of Medela. All rights reserved.
©2002 Medela, Inc.