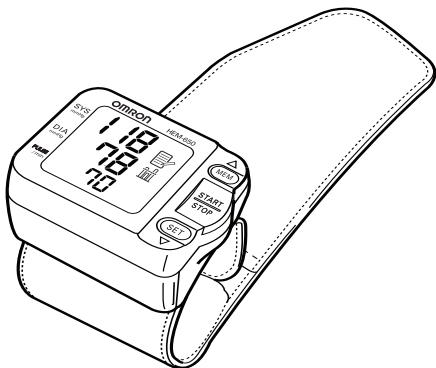
OMRON®

# Instruction Manual

## Automatic Blood Pressure Monitor with A.P.S.®

Model **HEM-650**

ESPAÑOL

ENGLISH

# TABLE OF CONTENTS

## Before Using the Monitor

Introduction .....	3
Important Safety Notes .....	4
Before Taking a Measurement .....	5

## Operating Instructions

Know Your Unit .....	6
Unit Display .....	7
Irregular Heartbeat Symbol .....	7
Battery Installation .....	8
Advanced Positioning Sensor .....	9
Position Sensor Alarm .....	9
Wrist Measurement Selection .....	10
Setting the Date and Time .....	11
Applying the Wrist Cuff .....	13
Taking a Measurement .....	14
Using the Memory Function .....	16

## Care and Maintenance

Care and Maintenance .....	18
Error Indicators and Troubleshooting Tips .....	19
FCC Statement .....	21
Warranty Information .....	22
Specifications .....	23

# INTRODUCTION

Thank you for purchasing the OMRON® HEM-650 Automatic Wrist Blood Pressure Monitor with A.P.S.®.

Fill-in for future reference

DATE PURCHASED: \_\_\_\_\_

SERIAL NUMBER: \_\_\_\_\_

Staple your purchase receipt here.

The monitor is compact and easy-to-use at home, at work, and during travel. Perfect for people who frequently monitor their own blood pressure.

OMRON® digital blood pressure monitors use the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through your artery and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use.

Clinical research has proven a direct relationship between blood pressure in the wrist and blood pressure in the arm. Changes in wrist blood pressure reflect changes in arm blood pressure because the arteries in the wrist and the arm are close to each other. Frequently measuring the blood pressure in your wrist will provide your physician with an accurate indication of changes in your blood pressure.

The OMRON® HEM-650 Automatic Wrist Blood Pressure Monitor with A.P.S.® is intended for home use.

**SAVE THESE INSTRUCTIONS**

## IMPORTANT SAFETY NOTES

**To assure the correct use of the product basic safety measures should always be followed including the precautions listed below:**

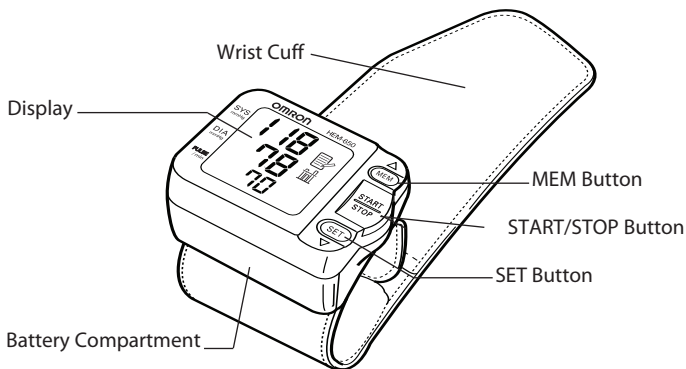
- ☑ Consult your physician before measuring blood pressure at the wrist if you have any of the following conditions: severe hypertension, severe diabetes, severe arteriosclerosis, severe kidney disease, or vascular disease that may compromise circulation.
- ☑ Read all information in the instruction book and any other literature included in the box before using the unit.
- ☑ Contact your physician for specific information about your blood pressure. Follow the instructions of your healthcare provider.
- ☑ Operate the unit only as intended. Do not use for any other purpose.
- ☑ The unit is intended for use in measuring blood pressure and pulse rate in adult patient population. Do not use on infants.
- ☑ Do not use a cellular phone near the unit. It may result in operational failure.
- ☑ Changes or modifications not approved by Omron Healthcare will void the user warranty. Do not disassemble or attempt to repair the unit or components.

## BEFORE TAKING A MEASUREMENT

1. Avoid eating, drinking alcohol, smoking, exercising and bathing for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. The cuff can be applied to your left or right wrist. Always measure your blood pressure on the same wrist.
4. Measurements should be taken in a quiet place.
5. Position the unit at heart level throughout the measurement.
6. Remain still and do not talk during the measurement.
7. Keep a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.
8. Wait 2-3 minutes between measurements. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement. You may need to increase the wait time depending on your individual physiological characteristics.

# KNOW YOUR UNIT


## Main Unit

## Accessories

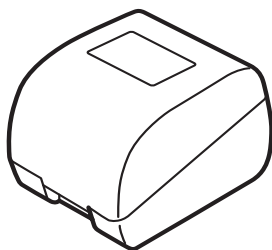
Two "AAA" Alkaline Batteries

Storage Case


# UNIT DISPLAY
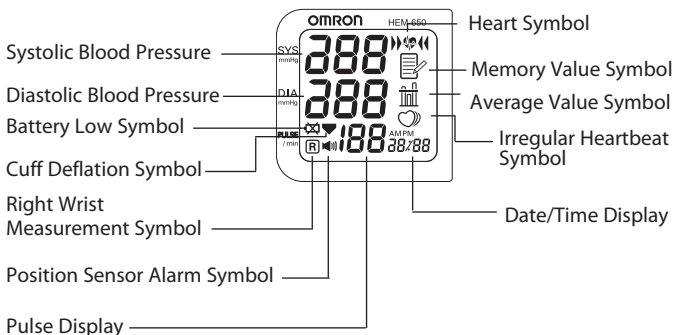
## IRREGULAR HEARTBEAT SYMBOL ( )

When the monitor detects an irregular rhythm two or more times during the measurement, the Irregular Heartbeat Symbol will appear on the display with the measurement values.

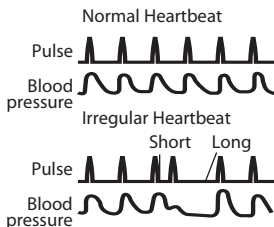An irregular heartbeat rhythm is defined as a rhythm that varies by less than 25% from the average rhythm or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure.


If the Irregular Heartbeat Symbol (  ) displays with your measurement results we recommend you consult your physician. Follow the directions of your doctor.

# BATTERY INSTALLATION

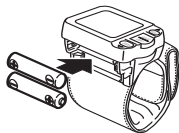
1. Press both sides of the battery cover and lift the cover off in the direction of the arrow.

2. Install two “AAA” Alkaline Batteries in the battery compartment, so that + (positive) and - (negative) polarities match the polarities of the battery compartment as indicated.

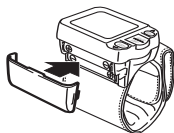3. Replace the battery cover. Make sure the battery cover is securely in place.

## BATTERY REPLACEMENT

When the Low Battery Symbol appears on the display screen remove all the batteries. Replace with two new batteries at the same time. When the Low Battery Symbol blinks a brief period of usage remains. Long life alkaline batteries are recommended.

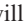
### NOTES:

- Measurement values stored in the memory will not be deleted during battery replacement.
- After replacing the batteries, reset the date and time. Refer to page 11, Setting the Date and Time.
- Dispose of batteries according to applicable local regulations.

## ADVANCED POSITIONING SENSOR

The Advanced Positioning Sensor, A.P.S.<sup>®</sup> will assure you that the unit is in the correct position before it starts to take a measurement.

1. Press the SET button while the power is turned off. The Position Sensor symbol (  ) will appear on the display.

2. Press the MEM button to select “on” or “oFF”. Press the START/STOP button to save this selection. When the feature is “on” the symbols will appear on the display. The heart symbol will begin to flash when your wrist is in the correct position.

## POSITION SENSOR ALARM

The Sensor Alarm emits sound to help you place your wrist in the correct position.

1. Press the SET button twice while the power is turned off. The Position Sensor Alarm symbol (  ) will appear on the display.

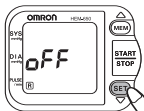
The alarm set “on 1” emits a series of blips if your wrist is too far away from the correct measuring position and beeps slowly once your wrist is in the correct position. When “on 2” is set the alarm only beeps once your wrist is in the correct position.

2. Press the MEM button to select “on 1”, “on 2” or “oFF”. Press the START/STOP button to save this selection.

## WRIST MEASUREMENT SELECTION

Blood Pressure Measurements can be taken on the left or right wrist.

1. Press the SET button three (3) times while the power is turned off. The Right Wrist Measurement symbol ( **R** ) will appear on the display.

Select “on” to measure your blood pressure using your right wrist.

Select “oFF” to measure your blood pressure using your left wrist.

2. Press the MEM button to select “on” or “oFF”. Press the START/STOP button to save this selection.


# SETTING THE DATE AND TIME

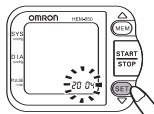The Blood Pressure Monitor stores up to 90 measurement values with the date and time. The monitor also calculates an average reading based on the last three measurement values.

Set the monitor to the current date and time before taking a measurement for the first time. If the current date and time is not set, the default date and time will display. The measurement values will be stored in the memory with the wrong date and time.

## 1. TO START THE SETTING

Press the SET button four (4) times while the power is turned off.  
The will year flash on the display.

## 2. SETTING THE YEAR

The year can be set between 2004 and 2030.  
When the display reaches 2030, it will return to 2004.

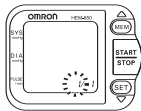
Press the MEM button to advance by increments of one year.

**NOTE:** Press and hold the MEM button to increase the values faster.

Press the SET button to set the current year.

The month flashes on the display.


# SETTING THE DATE AND TIME

## 3. SETTING THE MONTH

Press the MEM button to advance by increments of one month.

Press the SET button to set the current month.

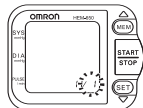The day flashes on the display.


## 4. SETTING THE DAY

Press the MEM button to advance by increments of one day.

Press the SET button to set the current day.

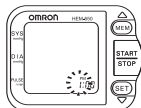The hour flashes on the display.

## 5. SETTING THE HOUR

The time is set using AM or PM.

Press the MEM button to advance by increments of one hour.

Press the SET button to set the current hour.

The minutes flash on the display.

## 6. SETTING THE MINUTE

Press the MEM button to advance by increments of one minute.

Press the SET button to set the current minute.

The year flashes on the display.

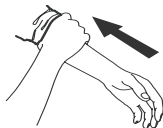
7. Press the START/STOP button to save this selection.

## APPLYING THE WRIST CUFF


Wrapping the wrist cuff is one of the most important points for taking a correct measurement.

### APPLYING THE CUFF ON THE LEFT WRIST

1. Roll up sleeve. Make sure your sleeve is not rolled up too tightly on your arm. This may constrict the flow of blood in your arm.
2. Wrap the cuff directly against your skin. Do not apply over clothing. Place the cuff over your left wrist with your left thumb facing upward.
3. Position the cuff leaving a clearance of approximately 1/2 to 1 inch between the cuff and the bottom of your palm.

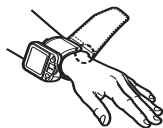4. Hold the bottom part of the cuff and wrap it around the wrist so it fits comfortably and securely around your wrist.


**NOTE:** Make sure the wrist cuff does not cover the protruding part of the wrist bone on the outside of the wrist.

5. Fold the remaining part of the wrist cuff back out of the way.

### APPLYING THE CUFF ON THE RIGHT WRIST

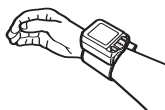When taking a measurement using the right wrist position the cuff as shown in the illustration.

Make sure the right wrist measurement symbol is displayed. Refer to page 10, Wrist Measurement Selection to select “on”.

## TAKING A MEASUREMENT

1. Sit comfortably in a chair with your feet flat on the floor.

2. Press the START/STOP button.

3. Hold your arm across your chest at heart level. Hold your elbow firmly to avoid body movement. Sit still and do not talk or move during the measurement.

4. Description of position sensor activity during measurement.

Adjust the height of your wrist until the position sensor alarm beeps slowly. When the monitor senses that your arm is in the correct position, the wrist cuff will automatically start to inflate and measurement starts.

**NOTE:** If your wrist is too low or too high, the position sensor alarm will emit a series of two short beeps similar to a heartbeat.

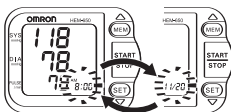
The current time is displayed and the Heart Symbol starts to flash while the cuff is inflating.

Hold the monitor at heart level until the measurement is completed.

**NOTE:** To stop the inflation or measurement, push the START/STOP button. The monitor will stop inflating, start deflating, and will turn off.

## TAKING A MEASUREMENT

- After the monitor has detected your blood pressure and pulse rate, the cuff automatically deflates. Your blood pressure and pulse rate are displayed.

Alternating Display

**NOTE:** The time and date of the measurement are displayed alternately.

- Press the START/STOP button** to turn the monitor off.

**NOTE:** The monitor will automatically turn off after two minutes.

### IRREGULAR HEARTBEAT FEATURE

The irregular heartbeat feature automatically determines if the measurement is usable.

If the measurement results are affected by irregular heartbeats but the result is valid, the result is shown together with the Irregular Heartbeat Symbol (  ).

If the irregular heartbeats cause the measurement to be invalid, no result is shown. If the Irregular Heartbeat Symbol (  ) is shown after you have taken a measurement, repeat the measurement.

If the Irregular Heartbeat Symbol (  ) is shown frequently, please notify your doctor.

## USING THE MEMORY FUNCTION

Every time you complete a measurement the monitor stores the blood pressure and pulse rate in the memory. The Blood Pressure Monitor automatically stores up to 90 measurement values. When 90 sets of measurement values are stored in the memory, the oldest record is deleted to save the most recent measurement values.

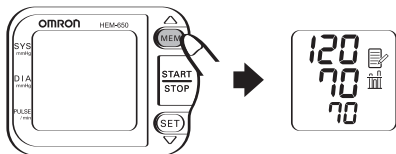
The monitor also calculates an average reading based on the values of the three most recent measurements taken. If the date and time is not set, the measurement values will not be stored in the memory correctly.

### TO DISPLAY THE AVERAGE VALUE

Average Value Symbol

1. **Press the MEM button.** The average value symbol will display on the screen with the average reading.


# USING THE MEMORY FUNCTION


## TO DISPLAY THE MEASUREMENT VALUES

The date and time is alternately displayed with the measurement values.

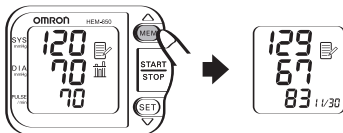1. **Press the MEM button** to display the measurement values stored in the memory beginning with the most recent values to the oldest values.

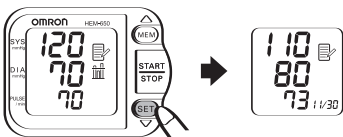**Press the SET button** to display the measurement values stored in the memory beginning with the oldest values to the most recent values.

### NOTES:

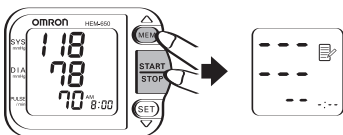
- Press the button repeatedly to display the next values.
- Press and hold the button to display the values faster.

2. **Press the START/STOP button** to turn the power off.

## TO DELETE ALL VALUES STORED IN THE MEMORY

You cannot partially delete records stored in the memory. All records will be deleted.

1. **Press and hold the MEM button and START/STOP button** simultaneously for more than 2 seconds.

## CARE AND MAINTENANCE

To keep your Automatic Wrist Blood Pressure Monitor in the best condition and protect the unit from damage follow the directions listed below:

**Clean the monitor with a soft dry cloth.** Do not use any abrasive or volatile cleaners. Do not attempt to wash the cuff. Never immerse the monitor or any components in water.

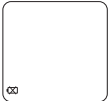
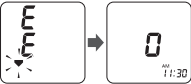
**Store the monitor in a safe and dry location.** Do not fold the cuff tightly. Do not subject the monitor to extreme hot or cold temperatures, humidity and direct sunlight. Store the monitor in the storage case when not in use.

**Avoid subjecting the monitor to strong shocks or vibrations,** such as dropping the unit on the floor.

**Remove the batteries** if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

**Use the unit consistent with the instruction provided in this manual.** Use only authorized parts and accessories.

## ERROR INDICATORS

SYMBOL	CAUSE	CORRECTION
 <p>After several seconds</p>	<p>Movement during measurement.</p>	<p>Press the START/STOP button to turn the unit off. Remove the monitor from your wrist. Refer to “Taking a Measurement” on page 14.</p> <p>Wait 2-3 minutes. Take another measurement.</p>
	<p>The cuff is not fastened securely.</p>	<p>Remove the monitor from your wrist. Refer to “Applying the Wrist Cuff” on page 13.</p> <p>Wait 2-3 minutes. Take another measurement.</p>
	<p>Wrist is not in the correct position.</p>	<p>Refer to “Taking a Measurement” on page 14. Make sure the wrist measurement selection is correct. Refer to page 10.</p>
	<p>Batteries are worn.</p>	<p>Replace the two “AAA” batteries. Refer to “Battery Installation” on page 8.</p>
 <p>After several seconds</p>	<p>Cuff over-inflated.</p>	<p>Repair service required. Refer to the warranty information on page 22.</p>

## ERROR INDICATORS

SYMBOL	CAUSE	CORRECTION
	Abnormal unit function.	Repair service required. Refer to “warranty information” on page 22.

## TROUBLESHOOTING TIPS

PROBLEM	CAUSE
No display appears on the unit when the START/STOP button is pressed.	Replace worn batteries. Check the battery installation for proper placement of the battery polarities.
Measurements values appear too high or too low.	Blood pressure reading varies constantly. Many factors including stress, time of day, how you apply the cuff, may affect your blood pressure reading. Review the sections “Before Taking a Measurement”, “Applying the Wrist Cuff” and “Taking a Measurement”.

## FCC STATEMENT

### **NOTE:**

**POTENTIAL FOR RADIO/TELEVISION INTERFERENCE** (for U.S.A. only) This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### **POTENTIAL FOR RADIO/TELEVISION INTERFERENCE** (for Canada only)

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Changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

## WARRANTY INFORMATION

Your HEM-650 Automatic IntelliSense® Wrist Blood Pressure Monitor with A.P.S.®, excluding the monitor cuff, is warranted to be free from defects in materials and workmanship appearing within 5 years from the date of purchase, when used in accordance with the instructions provided with the monitor. The monitor cuff is warranted to be free from defects in materials and workmanship appearing within one year from the date of purchase when the monitor is used in accordance with the instructions provided with the monitor. The above warranties extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor or monitor cuff covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service contact Omron Healthcare's Customer Service by calling **1-800-634-4350** for the address of the repair location and the return shipping and handling fee. Information for warranty service is available on our website at [www.omronhealthcare.com](http://www.omronhealthcare.com).

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

**ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.**

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.** Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

### **FOR CUSTOMER SERVICE (U.S. & Canada):**

Visit our website:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

## SPECIFICATIONS

Name:	<b>OMRON</b> Automatic Wrist Blood Pressure Monitor with A.P.S. <sup>®</sup>
Model:	<b>HEM-650</b>
Display:	Digital Display
Measurement:	Oscillometric method
Measurement Range:	Pressure; 0 to 299 mmHg Pulse; 40 to 180 beats/min
Memory:	90 measurements with date and time
Accuracy:	Pressure; Within $\pm 3$ mmHg Pulse rate; Within $\pm 5\%$ of reading
Inflation:	Automatic inflation by pump
Deflation:	Automatic rapid deflation
Pressure Detection:	Electrostatic capacity semiconductor pressure sensor
Power supply:	Two "AAA" batteries
Battery Life:	Approximately 300 measurements when using alkaline batteries at the room temperature of 73°F (23°C) and by using three times a day and inflating to 170 mmHg
Operating Temperature and Humidity:	50°F to 104°F (10°C to 40°C), 30 to 85% RH
Storage Temperature and Humidity:	-4°F to 140°F (-20°C to 60°C), 10 to 95% RH
Weight of Main Unit:	Approximately 110 g (not including batteries)
External Dimensions:	2 3/4" (w) x 2 1/8" (h) x 1 7/16" (d) 70 mm x 54 mm x 37 mm (not including the wrist cuff)
Measurable Circumference of Wrist:	Approximately 5 1/4" to 8 1/2" (Approximately 135mm to 215mm)
Electric Shock Protection:	Internal power supply appliance type B
Contents:	Storage case, two "AAA" alkaline batteries, main unit, instruction manual
UPC:	0 73796 65036 0

These specifications, to improve performance, are subject to change without notice.

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Copyright © 2005 OMRON HEALTHCARE, INC.
**Omron Healthcare, Inc.**