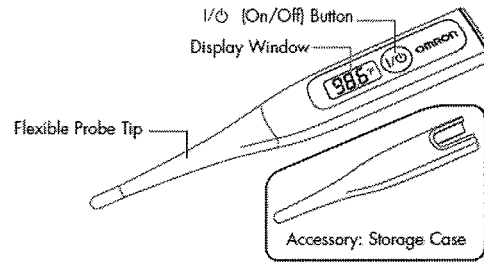
# OMRON®

## 10 SECOND FLEX

### DIGITAL THERMOMETER

#### Model MC-343

9486926-3A

Thank you for purchasing an Omron 10 Second Flex Digital Thermometer.

This thermometer offers a safe, accurate and quick temperature reading. There are three ways to take a temperature; oral, rectal or underarm. This thermometer eliminates any worries about broken glass or mercury hazards. Note that your temperature is affected by smoking, eating or drinking. We recommend that you practice taking a temperature on each member of your family when they are healthy. This way you will be confident in your technique when using the thermometer on a sick family member. The enclosed MC-343 thermometer is not sterile.

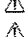
Read this Instruction Manual completely before using your digital thermometer.

If you have questions regarding a specific temperature, please contact your physician.

## SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed below.

SAFETY SYMBOLS USED IN THIS INSTRUCTION SHEET	
 <b>WARNING</b>	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 <b>CAUTION</b>	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

-  Contact your physician for specific information about your temperature readings. Self-diagnosis of measured results and treatment are dangerous. Follow the instructions of your physician or licensed healthcare provider.
-  Adult supervision is required when taking a temperature on children. Unattended children may injure themselves.
-  Store the thermometer out of the reach of children. The thermometer contains small parts that may cause a choking hazard if swallowed.
-  Do not take a measurement when the thermometer is wet. This may result in an inaccurate reading.
-  This thermometer is intended for use in measuring temperature in the human body by oral, underarm and rectal methods. Operate the thermometer only as intended. Do not use the thermometer for any other purpose.
-  Read all of the information in the instruction sheet before using the thermometer.
-  Do not use the thermometer to take a rectal reading on persons with rectal disorders. This may worsen the person's condition.
-  When taking a rectal reading, do not force the thermometer into the rectum. Immediately stop inserting the thermometer if any pain is present. Failure to do this may cause an injury.
-  Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique. Avoid taking a reading after exercise, bathing, eating and drinking for 30 minutes.
-  When taking an oral reading, do not bite on the thermometer. This may cause the thermometer to break and cause an injury.
-  Do not use a mobile phone near the thermometer or in places where strong static electricity or electromagnetic fields are present. This may cause an inaccurate reading or operational failure.
-  Remove the battery if the thermometer will not be used for 3 months or longer.
-  Dispose of the battery according to applicable local regulations. Unlawful disposal may cause environmental pollution.
-  Use only Omron authorized parts and accessories. Parts and accessories not approved for use with the thermometer may damage the thermometer.
-  Do not attempt to disassemble or repair the thermometer. This may result in inaccurate readings and will void the user warranty.
-  Do not subject the thermometer to strong shocks, such as dropping the thermometer on the floor. Do not step on the thermometer or storage case. Do not bend the tip more than 45 degrees.
-  Store the thermometer in a clean, safe location.
-  Do not attempt to incinerate the battery. It may burst.
-  Pay attention to polarity (+-) when replacing the battery. Failure to do so may lead to fluid leakage, heat generation or bursting, resulting in damage to the unit.
-  Do not share the thermometer among individuals.

## ■ CORRECT MEASUREMENT

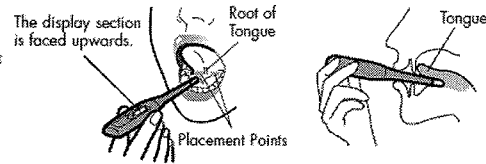
Please keep still during measurement.

Measurement accuracy cannot be assured when the method used to measure the temperature is incorrect.

### ORAL USE (for ages 4 and older):

The mouth should remain closed up to 5 minutes before attempting a reading.

- Place the thermometer in the mouth under the tongue so that it rests to the left or right of the root of the tongue.
- Use downward tongue pressure to hold the thermometer in place.
- Hold the thermometer to keep it from sliding around in the mouth.


### RECTAL USE:

Always use a probe cover when taking rectal measurements.


- Commonly used for young children when it is difficult to take an oral or underarm temperature.
- Lubricate with a water-soluble gel. Do not use petroleum jelly.
- Gently insert the probe tip no more than 1/2 inch into the rectum. Do not force the tip into the rectum if resistance is encountered.
- Disinfect the thermometer after use. Refer to Cleaning and disinfection in the "CARE AND MAINTENANCE" section.

### UNDERARM (AXILLARY) USE:

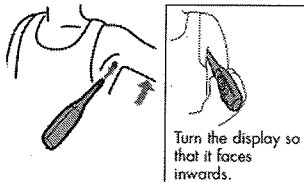- Place the probe tip in the center of the underarm.

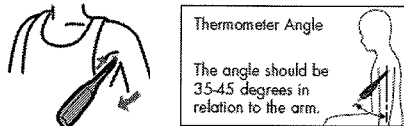- Lock the probe tip under the arm, using the arm to slightly apply pressure inwards.

\* In the case of infants and very young children, gently hold the arm to prevent movement.

### INCORRECT MEASUREMENTS

- Taking the temperature immediately after exercising, bathing, eating or drinking will result in an incorrect measurement.  
Correct: Wait at least 30 minutes before taking the temperature.
  - Taking the temperature after moving around.  
Correct: Take the temperature immediately after waking up or wait for at least 30 minutes after moving around.
  - Moving around after awakening increases the body temperature.
- Only applies when taking underarm temperatures -**  
The following may result in incorrect measurements.
- Heavy underarm perspiration
  - Taking the temperature after being under the blankets for a long time.  
Correct: Wipe off any underarm perspiration before taking the temperature.

## ■ TAKING THE TEMPERATURE

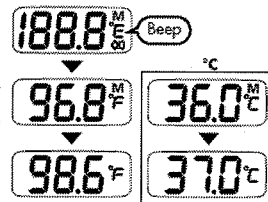
The use of a new probe cover is recommended for each use to avoid spreading germs that may lead to bacterial or viral infection.

### 1. Press the 1/0 button.

A default temperature 98.6°F (37.0°C) is shown on display.

#### NOTE:

When a temperature other than 98.6°F (37.0°C) is displayed, please refer to the "TROUBLESHOOTING" section for details to correct the problem. If a previous temperature was taken, it will display before taking the current temperature.

All of the display symbols appear.

The most recently measured temperature is displayed together with the M indicator.


Waiting for use.

### 2. Apply the thermometer to the measurement site.

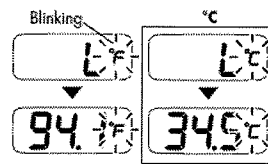Use the oral, rectal or underarm site for measurement. Remain still during the measurement.

#### NOTE:


Even if the "L" indicator fails to display (when only the temperature is displayed), this is not an error. So the temperature may be taken as is.

When the [F/°C] indicator symbol blinks, the thermometer is ready for use.

Temperature measurement will commence and the [F/°C] indicator symbol will blink.

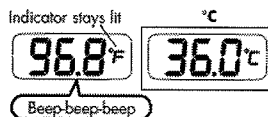- ### 3. When the reading is ready, the thermometer will emit a [beep-beep-beep] sound three times. Remove the thermometer from the measurement site and read the result.

The [F/°C] indicator symbol will stop blinking and stay on when the reading is ready.

### BUZZER NOTIFICATION TIMING

**Oral : approx. 20 seconds**

**Rectal : approx. 10 seconds**

**Underarm : approx. 25 seconds**

#### NOTES:

- The buzzer notification timing is based on an environmental temperature of 73.4°F (23°C) without the probe cover.
- There will be individual differences in the measurement results.

- ### 4. Turn off the thermometer and store it in the storage case.

#### NOTES:

- The reading will automatically be stored in memory.
- Although the thermometer will automatically switch itself off 30 minutes after use or after 3 minutes when it has not been used; it should be turned off manually to conserve the battery.

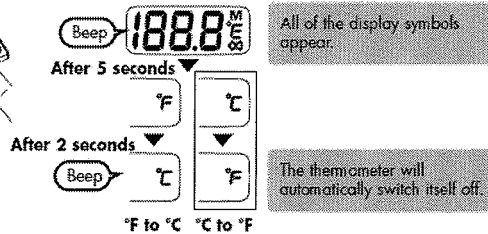
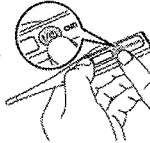
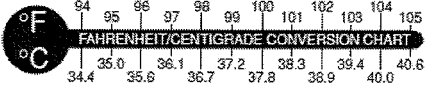
## ■ DISPLAYING THE LAST TEMPERATURE FROM MEMORY RECALL

Press the  $\text{M} \text{E} \text{C}$  button to display the temperature that was measured most recently. The last temperature will quickly display before going back to the default setting of 98.6°F (37.0°C).  
For details, please refer to the "TAKING THE TEMPERATURE" section.

## ■ SWITCHING BETWEEN FAHRENHEIT/CENTIGRADE (CELSIUS)

Press and hold the  $\text{M} \text{E} \text{C}$  button for 7 seconds until a beep is emitted.

The display will change to the other measurement scale at the time when the beep sounds.


## ■ BATTERY REPLACEMENT

**Battery:** Alkaline-Manganese Button Battery LR41 (available commercially)


### Battery Replacement Indicator ( $\text{X}$ )

Replace the battery when the battery replacement indicator appears when the thermometer is switched on.

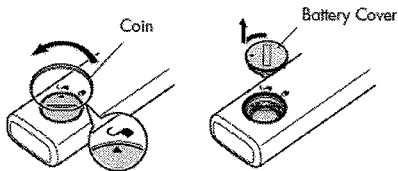
1. Use a coin to take off the battery cover.

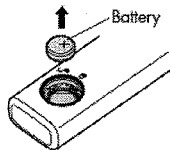
2. Remove the battery.

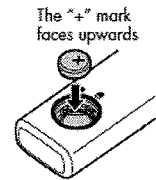3. Insert the new battery with the "+" pole facing up as shown in the diagram.

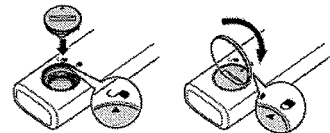4. Use a coin to replace the battery cover.

### NOTES:

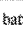
- The thermometer comes fitted with a battery for testing purposes. This battery may not provide the same amount of battery life as a new battery.
- Consult your local authorities and follow their instructions for the disposal of batteries and/or the thermometer.
- Do not lose the O-ring around the battery cover. Water-resistant will fail and can lead to misreading and failure of the unit.

## ■ ERROR DISPLAYS

INDICATOR	CAUSE	REMEDY
L	Displayed when the temperature of the sensing section is less than 89.6°F (32°C).	This is not an error so it may be ignored and the temperature taken.
H	Displayed when the temperature of the sensing section is greater than 107.6°F (42°C).	Cool the sensing section and then take the temperature.
Temperature Displayed	Displayed when the surrounding temperature exceeds 89.6°F (32°C).	This is not an error, so the temperature may be taken as is.

## ■ TROUBLESHOOTING

PROBLEM	CAUSE	CORRECTION
Wide differences in readings	<ul style="list-style-type: none"> <li>The temperature sensing part of the thermometer is being placed at a different location for each reading.</li> <li>The thermometer is moved while the temperature is being taken.</li> <li>The mouth is kept open while the measurement is in progress.</li> </ul>	Check to see that the thermometer is being placed at the location as described in the "CORRECT MEASUREMENT" section.
Nothing appears in the display section when the power switch is pressed.	The polarity ⊕ ⊖ is reversed.	Take out the battery and replace using the correct polarity.
	The battery is depleted.	Check to see if  appears in the display section. Replace with a new battery (LR41).
[98.6°F (37.0°C)] is not displayed during the measurement preparation period.	A temperature from [98.4°F (36.9°C)] to [98.8°F (37.1°C)] is displayed.	The thermometer may be used as it is.
	A temperature other than that in the range from [98.4°F (36.9°C)] to [98.8°F (37.1°C)] is displayed, even after turning the thermometer on and off several times.	Please contact your nearest authorized OMRON dealer.

## ■ CARE AND MAINTENANCE

### Keep the thermometer clean.

- Do not use strong jets of water to clean the thermometer or leave it immersed in water for long periods of time.
- Wipe the thermometer with a soft clean cloth.
- When using alcohol to clean the thermometer, make sure that it does not come into contact with the indicator section.
- For stubborn stains, wipe the thermometer with a cloth that has been dampened with water or a neutral detergent solution and then wring thoroughly. Finish by wiping with a soft dry cloth.
- Observe the following to prevent damage to the thermometer.
  - Do not use benzene, thinner, gasoline or other strong solvents to clean the thermometer.
  - Do not soak the sensing section in alcohol for long periods of time or attempt to sterilize it using hot water (water at a temperature of 122°F (50°C) or higher).
  - Do not use ultrasonic washing to clean the thermometer.

### Store the thermometer in its storage case.

- Do not store the thermometer in the following types of places. Doing so may damage the thermometer.
  - Wet locations.
  - Locations with high heat and humidity or those that are exposed to direct sunlight. Areas close to heating equipment, dusty locations, or environments where there are high salt concentrations in the air.
  - Locations where the unit will be subjected to leaning, shock or vibration.
  - Pharmaceutical storage areas or locations where corrosive gases are present.

### Cleaning and disinfection

- Omron recommends to wipe the device with damp cloth. For disinfection, 70% Ethanol or Isopropyl alcohol can be used. Do not attempt to disinfect the sensing section of the thermometer by immersing in alcohol or in hot water (water over 122°F (50°C)).

## ■ LIMITED WARRANTIES

Your new digital thermometer, excluding the battery, is guaranteed to be free from manufacturers defects for one year under normal use. This warranty extends only to the original retail purchaser. We will, at our option, repair or replace without charge any thermometer. To obtain warranty service contact Customer Service by calling 1-800-634-4350 for the address location and the return shipping and handling fee.

THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

## ■ SPECIFICATIONS

Product Name:	10 SECOND FLEX DIGITAL THERMOMETER MC-343 (MC-343F-Z)	Power Supply:	1.5V DC, 1 LR41 Alkaline-Manganese Button Battery
Power Consumption:	0.1mW	Battery Life:	Approx. 2 years or more (3 times per day)
Sensing Unit:	Thermistor	Measurement Method:	Actual measurement
Temperature Display:	4-digit (°F), or 3-digit (°C) display in 0.1 degree increments	Applied Part:	 = Type E
Measurement Accuracy:	± 0.2°F (89.6 to 107.6°F) ± 0.1°C (32.0 to 42.0°C) (when measured at a standard room temperature of 73.4°F (23°C) in a test tank held at constant temperature)		
Measurement Range:	89.6 to 107.6°F (32.0 to 42.0°C)		
Usage Environment	Surrounding temperature +50 to +104°F (+10 to +40°C), Relative Humidity 30-85%RH		
Temp and Humidity:			
Storage Environment	Surrounding temperature -4 to +140°F (-20 to +60°C), Relative Humidity 10-95%RH		
Temp and Humidity:			
Weight:	Approx. 0.42 OZ (12g) with battery installed		
External Dimensions:	0.76"(w) × 5.22"(l) × 0.39"(d)		
Contents:	Thermometer, Battery (LR41), Storage Case, Instruction Sheet, 5 Probe Covers		

This thermometer is intended for home use.

The specifications may be changed without prior notice.

Manufactured by OMRON HEALTHCARE Co., Ltd.  
Made in China

Distributed by :  
OMRON HEALTHCARE, INC.  
1200 Lakeside Drive  
Bannockburn, Illinois 60015

Toll Free Customer Service:  
1-800-634-4350  
www.omronhealthcare.com  
Copyright © 2010 Omron Healthcare, Inc.

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