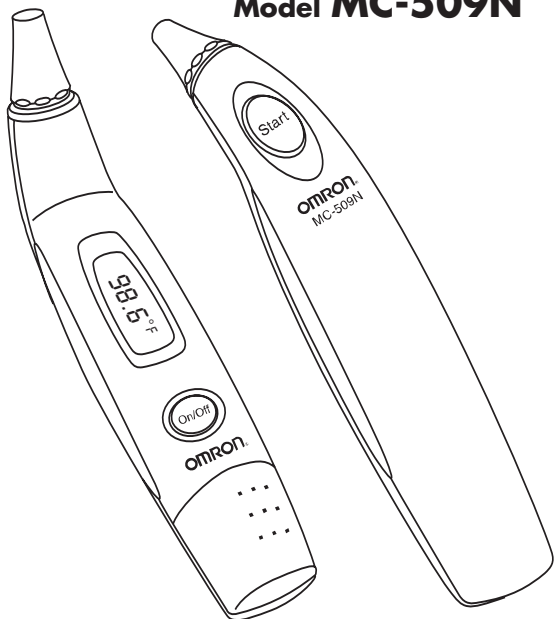
# OMRON®

## Instruction Manual

# Instant Ear Thermometer

Model **MC-509N**

ESPAÑOL

ENGLISH

# Table of Contents

Introduction .....	3
Warnings .....	4
Part Names .....	5
Conversion °F to °C .....	5
Before Use .....	6
Directions for Use .....	7-8
Battery Replacement .....	8
Care and Maintenance .....	8
Trouble Shooting .....	9
Error Messages .....	10
Specifications .....	11
Warranty .....	12

# Introduction

Thank you for purchasing the Omron® Instant Ear Thermometer.

Please read carefully this instruction manual completely before using the thermometer.

This ear thermometer offers a safe, accurate and quick temperature reading from the ear canal. This thermometer senses the infrared heat from the eardrum (tympanic membrane).

No need to worry about broken glass or mercury hazards.

An ear temperature is not affected by talking, eating or drinking. These factors do affect an oral temperature.

In one second, it changes the heat measured into an equivalent oral temperature.

The unit is easy to use for children and adults. Temperatures can be taken when a child is sleeping.

We recommend you practice taking a temperature with the Omron® Instant Thermometer on each member of your family when they are healthy. This way you will be confident in your technique when using the ear thermometer on a sick family member.

The Omron® Instant Ear Thermometer has been clinically tested and proven to be safe and accurate when used in accordance with this operating instruction manual.

Contact your Healthcare Provider if you have any questions regarding specific temperatures.

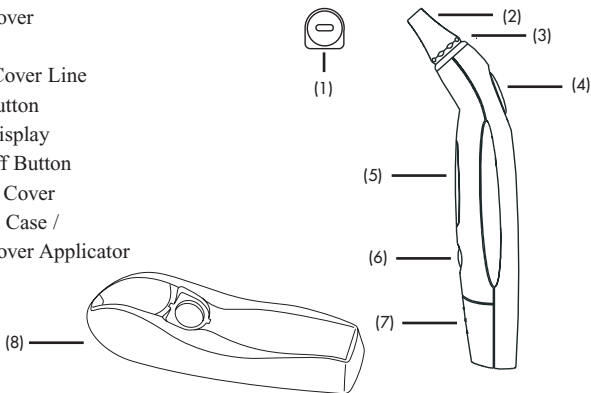
*This product is intended for home use only.*

# Warnings

- Use of this thermometer is **not intended** as a substitution for consultation with your physician.
- This thermometer is intended to be used in human ears only.
- Use only Omron® lens covers for this specific unit.
- For proper hygiene, **DO NOT** reuse the same lens covers.
- Replace the lens cover when it is affected by earwax or other body fluids.
- Earwax in the ear canal may cause a lower temperature reading. Make sure patient's ear canal is clean to ensure an accurate reading.
- Handle the thermometer with care. Improper handling may affect temperature accuracy.
- Keep the unit and lens covers out of the reach of children.
- Use your unit in room temperature. Extremes in environmental temperature may affect the accuracy.
- Store unit at room temperature when possible. Keep thermometer at room temperature for at least 30 minutes before use.
- Remove unit if pain is noted during use.
- **DO NOT** use if ear infection is present.
- **DO NOT** use right after swimming or bathing as this may injure ear tissue as well as affect the accuracy of the temperature.
- **DO NOT** take apart, repair or change any parts of the unit at any time.
- **DO NOT** get the unit wet or pour any liquid onto any portion of the unit.


# Part Names

- (1) Lens Cover
- (2) Probe
- (3) Probe Cover Line
- (4) Start Button
- (5) LCD Display
- (6) On / Off Button
- (7) Battery Cover
- (8) Storage Case /  
Lens Cover Applicator

## Conversion Between Fahrenheit and Celsius (°F/°C)

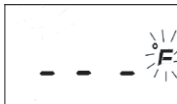- The thermometer is preset at (Fahrenheit) °F.
- Follow the steps below to change the measurement scale from (Fahrenheit) °F to (Celsius) °C.
  - 1) Turn off the unit, press and hold the start button.
  - 2) Release the button after approximately 5 seconds when the (Celsius) °C or (Fahrenheit) °F symbol appears on the LCD display.
  - 3) Press the Start button again until desired measurement scale is reached.
  - 4) When measurement scale has been chosen, wait 5 seconds and the unit will enter the ready for measurement mode automatically.
- Repeat the process to reverse measurement scale if desired.


# Before Use


**Important:** Prior to every measurement, fit a new lens cover on the probe. Failure to do so may result in incorrect temperature measurement. For hygienic use, attach a new lens cover each time.

## Replace Lens Cover

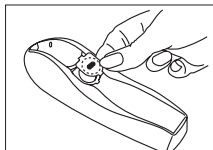
- Use only Omron® Lens Covers (model MC-509NLC)

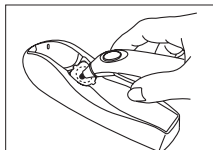
1) Place a lens cover onto storage case hole with paper side upwards (plastic side downwards).

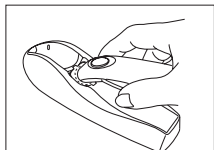2) Position the unit vertically and insert the probe into the center part of the lens cover.

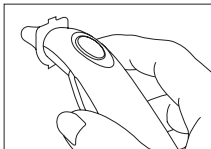3) Completely push the probe through the lens cover hole.


4) After feeling a slight click, remove the unit, and the lens cover will be attached.


**Please check that the lens cover is fitted firmly (over the Probe Cover Line on the Probe) before use. If the lens cover becomes damaged, replace with a new one immediately.**
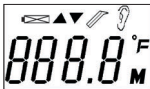
# Directions for Use

1. With lens cover correctly on unit, press On/Off button. The LCD display is activated to show all segments for 2 seconds (illuminated display for 5 seconds).

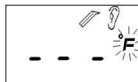2. The last measurement reading will be shown on the display automatically for 3 seconds with the "M" memory symbol.

3. When the °C or °F symbol is flashing, a beep sound is heard and the thermometer is ready for measurement.

4. Straighten the ear canal by pulling the ear up and back to give a clear view of the eardrum.

- For children under 1 year:  
Pull the ear straight back
- Children aged 1 year to adult:  
Pull the ear up and back


## NOTE:

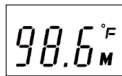- For an infant, it is best to have the child laying flat with his head sideways so the ear is facing upwards. For an older child or adult, it is best to stand behind and slightly to the side of the individual.

5. While tugging the ear, insert the probe snugly into the ear canal and press the "START" button. Release it, when you hear a beep sound. This is the reminding signal that confirms the end of measurement. After completing measurement, the display will illuminate and stay lit for 5 seconds.

6. Remove the thermometer from the ear canal. The LCD displays the measured temperature.
7. Replace the lens cover after each measurement. To do this, please follow the instructions in "Replace Lens Cover"

## 8. In order to assure accurate readings, PLEASE wait at least 30 seconds after 3-5 continuous measurements

9. Turn the unit off by pressing the On/Off button. The thermometer will automatically turn off in approximately 1 minute after a temperature has been taken.

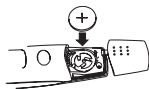
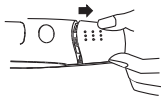
## Directions for Use (continued)

- In the following situations it is recommended that three temperatures in the same ear be taken and the highest one taken as the reading:
  - 1) New born infants in the first 100 days.
  - 2) Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
  - 3) When the user is learning how to use the ear thermometer for the first time until he/she has familiarized himself/herself with the instrument and obtains consistent readings.

## Battery Replacement

The thermometer is supplied with one lithium battery, type CR2032. If the battery symbol is displayed, replace battery at once. Remove the battery cover by sliding it in the direction shown. Take out the battery and put in a new one.

Battery Symbol

## Care and Maintenance

To keep your thermometer in the best condition and protect the unit from damage follow the directions listed below:

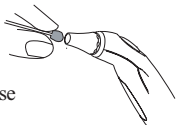
**Clean the unit with a soft dry cloth.** Do not use any abrasive or volatile cleaners. Rubbing alcohol may be applied to a soft cloth to wipe off the casing. To clean the lens wipe with a soft dry cloth or cotton swab. Never immerse the thermometer or any of the components in water.

**Store the thermometer in a safe and dry location.** After each use attach a new lens cover to protect the lens. When not in use keep the thermometer in the storage case. Do not subject to extreme hot or cold temperatures, humidity and direct sunlight. Avoid subjecting the thermometer to strong shocks such as dropping on the floor, vibration or dust.

**Use the unit consistent with the instruction provided in this manual.**

Use only authorized parts and accessories.

Download from [www.Somanuals.com](http://www.Somanuals.com). All Manuals Search And Download.




# Trouble Shooting

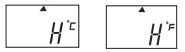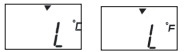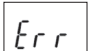LCD Display	Display Meaning	Action
	Display of all segments	Press the On/Off button to run on the unit, all segments will be shown for 2 seconds; lighted display will be shown for 5 seconds.
	Recall of last reading	The last reading will be automatically shown on the display for 3 seconds.
	Ready for measurement	The unit is ready for measurement. The °C or °F symbol will keep flashing.
	Completion of a measurement	The reading will be shown on the LCD display. The unit is ready for the next measurement when the °C or °F symbol starts flashing.
	Ready for next measurement	The °C (or °F) symbol appears and flashes to indicate readiness for the next measurement
	Low battery indication	The battery symbol keeps flashing when the unit is turned on to alert the user to replace the battery.

# Error Messages

Display/ Problem	Display Meaning	Cause/Action
	Measured temperature is too high.	Displays "H" when measured temperature is higher than 42.2°C or 108.°F.
	Measured temperature is too low.	Displays "L" when measured temperature is lower than 32.0°C or 89.6°F.
	Ambient temperature too high.	Displays "H" in conjunction with the "▲" when ambient temperature is higher than 40.0 °C or 104.0 °F.
	Ambient temperature too low.	Displays "L" in conjunction with the "▼" when ambient temperature is lower than 10.0 °C or 50.0 °F.
	Error function display.	When system malfunctions.
	Blank display.	Please check if the battery has been installed correctly. Also check polarity (<+> and <->) of battery.
	Low battery indication.	If the steady battery symbol is the only symbol shown on the display, the battery should be replaced immediately.

# Specifications

Name:	OMRON® Instant Ear Thermometer
Model:	MC-509N
Measuring Range:	32.0 °C to 42.2 °C (89.6 °F to 108.0 °F)
Accuracy:	Laboratory: ±0.2 °C, 32.0 ~ 42.2 °C (±0.4 °F, 89.6 ~ 108.0 °F)
Operating Temperature:	10.0 °C to 40.0 °C (50.0 °F to 104.0 °F)
Storage Temperature:	-20.0 °C to 50.0 °C (-4.0 °F to 122.0 °F) / 15% to 95% RH non condensing
Display:	Liquid Crystal Display with resolution 0.1 °C (0.1 °F)
Response Time:	Approx. 1 second
Acoustic:	a. The unit is turned ON and ready for the measurement: 1 short beep sound. b. Complete the measurement: 1 long beep sound. c. System error or malfunction: 3 short beeps.
Memory:	Displays the last measured temperature for 3 seconds when unit is turned on.
Automatic Switch-off:	Approx. 1 minute after last measurement is completed.
Battery:	One 3 volt battery, CR2032 (approx. 1,500 measurements)
Low Battery Indication:	Replace the battery when the low battery symbol  is shown on the LCD.
Dimensions:	6" x 1 1/8" x 1 1/4" ; (H) 153mm (L) x 31mm (W) x 45mm
Weight:	1.9oz. / 53g (with battery), 1.8oz. / 50g (w/o battery)
Standards:	Complies with ASTM E-1965-98 standards.
Components:	Storage Case (with built in lens cover applicator) 20 Lens Covers: type MC-509NLC, one-3volt Lithium Battery: type CR2032, and Instruction Manual
Replacement Lens Covers:	USE ONLY OMRON® Lens Covers model: MC-509NLC (40 ct.)

# Limited 3 Year Warranty

Your Omron® Instant Thermometer (MC-509N) is warranted to be free from manufacturing defects for a period of three years under normal use. This warranty extends only to the original retail purchaser. The three year warranty excludes the battery. Should repair be needed within the warranty period, ship the unit prepaid with proof of purchase date and \$5.00 for return shipping and insurance to:

Omron Healthcare Inc.  
300 Lakeview Parkway,  
Vernon Hills, Illinois 60061  
Attn: Service Dept.

Be sure to include the model number of your unit and your phone number on any correspondence. We will either repair or replace (at our option) free of charge any parts necessary to correct defects in the materials or workmanship.

The above warranty is complete and exclusive. The warrantor expressly disclaims liability for incidental, special, or consequential damages of any nature. (Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above warranty may not apply to you.)

Any implied warranties arising by the operation of law shall be limited in duration to the term of this warranty. (Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.)

This warranty gives you specific legal rights and you may have other rights which vary from state to state.

FOR CUSTOMER SERVICE CALL TOLL FREE:

1-800-634-4350

Distributed by:  
**OMRON HEALTHCARE, INC**  
300 Lakeview Parkway  
Vernon Hills, Illinois 60061  
[www.omronhealthcare.com](http://www.omronhealthcare.com)

Copyright © /2002 OMRON HEALTHCARE, INC.

509NINST Rev.

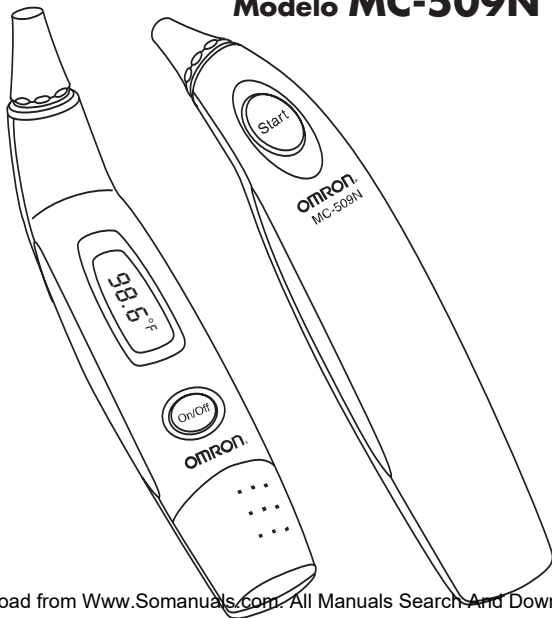
Download from [Www.Somanu12s.com](http://Www.Somanu12s.com). All Manuals Search And Download.

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Omron Healthcare, Inc.