Panasonic

Wrist Blood Pressure Monitor

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Model No. EW280

Modelo No. EW280

Before operating this device, please read these instructions completely.

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Introduction

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Thank you for purchasing the Panasonic Wrist Blood Pressure Monitor EW280. Designed for easy, one-touch operation, this monitor incorporates Precise Logic™ technology to provide accurate, reliable measurements. It comes with a handy carrying case that also serves as an armrest during measurement.

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Measuring your own blood pressure is an important way of monitoring your health. High blood pressure (hypertension) is a major health problem which can be treated effectively once detected. Measuring your blood pressure between doctor visits on a regular basis in the comfort of your home, and keeping a record of the measurements, will help you monitor any significant changes in your blood pressure. Keeping an accurate record of your blood pressure will help your doctor diagnose and possibly prevent any health problems in the future.

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Your heart acts like a pump, sending blood surging through your blood vessels each time it contracts. Blood pressure is the pressure exerted by blood pumped from the heart on the walls of blood vessels. Systolic pressure is the pressure exerted when the heart contracts and pumps blood into the arteries. Diastolic pressure is the pressure exerted when the heart expands, or relaxes. When you or your doctor take your blood pressure, both your systolic and diastolic pressures are measured. If your blood pressure measurement is 120 over 80 (120/80), for example, your systolic pressure is 120 while your diastolic pressure is 80.

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Important Instructions Before Use

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1. Do not confuse self-monitoring with self-diagnosis. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.

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2. If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.

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3. For persons with irregular or unstable circulation resulting from diabetes, liver disease, arteriosclerosis or other medical conditions, there may be variations in blood pressure values measured at the wrist versus at the upper arm. Monitoring the trends in your blood pressure taken at either the arm or the wrist is nevertheless useful and important.

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4. Blood pressure can vary based on many factors, including age, gender, weight and physical condition. In general, a person's blood pressure is lower during sleep and higher when he or she is active. Blood pressure can change easily in response to physiological changes. The setting in which a person's blood pressure is measured can also affect the results. Having one's blood pressure

measured by a healthcare professional in a hospital or clinic can cause nervousness and may result in a temporarily elevated reading. Because blood pressure measurements taken in a clinical setting can vary considerably from those taken at home, a person's blood pressure should be measured not only occasionally in the doctor's office, but also on a regular basis at home. Also, if you find that your blood pressure is lower at home, this is not unusual. To accurately compare with your physician's reading, take your Panasonic blood pressure monitor to your doctor's office and compare readings in this setting.

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5. People suffering from cardiac arrhythmia, vascular constriction, liver disorders or diabetes, people with cardiac pacemakers or a weak or irregular pulse, and women who are pregnant should consult their physician before measuring their blood pressure themselves. Different values may be obtained due to their condition.

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6. Try to take your blood pressure measurements at the same time and under the same conditions every day.

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- The ideal time to measure your blood pressure (to obtain your so-called "base blood pressure") is in the morning just after waking up, before having breakfast and before any major activity or exercise. If this is not possible, however, try to take measurements at a specified time prior to breakfast, and before you have become active. You should relax for about 5 minutes before taking the measurement.

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- The following situations may cause substantial variations in blood pressure readings and should therefore be avoided at least 30 minutes prior to taking your blood pressure.

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7. Measurements may be impaired if this device is used near a television, microwave oven, X-ray equipment or other devices with strong electrical fields. To prevent such interference, use the monitor at a sufficient distance from such devices or turn the devices off.

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8. This device is designed for use by adults. Consult with your physician before using this device on a child. Do not use on infants or toddlers.

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- 9. This device is not suitable for continuous monitoring during medical emergencies or operations.**

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- 10. Do not use the unit for any purpose other than measuring blood pressure.**

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- 11. Improper handling of batteries may result in battery explosion or in corrosion from battery leakage. Please observe the followings to ensure proper use of batteries.**

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- a. Be sure to turn off the power of the device after use.**

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- b. Do not mix different types or sizes of batteries.**

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- c. Replace all batteries at the same time. Do not mix old and new batteries.**

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d. Be sure to insert batteries with correct polarity, as instructed.

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e. Remove batteries when they are worn out, and dispose of them properly according to all applicable environmental regulations.

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f. Do not disassemble or throw batteries into a fire.

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g. Do not short-circuit batteries.

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


h. Do not charge the batteries included with this unit.

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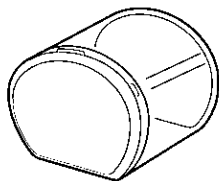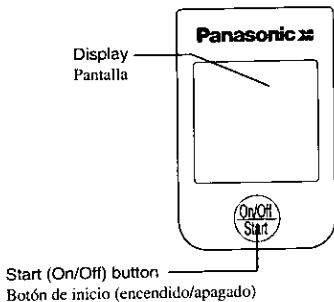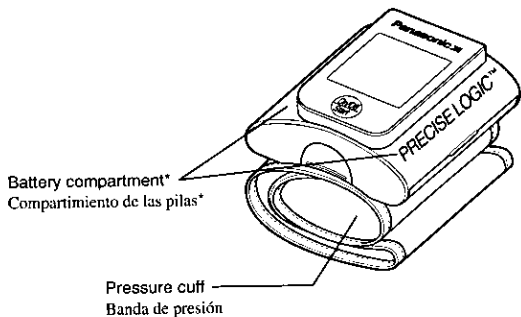Loading Batteries

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1. Grasp the battery compartment cover and remove it upwards as shown in the Fig. 2-a.

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2. Insert two 1.5 V alkaline (AAA/LR03) batteries, observing the (+) and (-) signs to ensure correct polarity.

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3. Replace the battery compartment cover until it clicks into place (Fig. 2-b).

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- Approximately 500 measurements can be taken with a new set of Panasonic alkaline batteries (at an ambient temperature of 22°C or 71.6°F, pressurization of 180 mmHg).

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
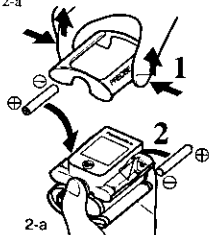
Fig. 2-b
Figura 2-b

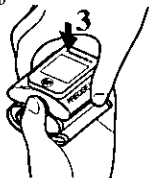
When to Replace Batteries

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Replace the batteries when the battery symbol Fig. 3 appears on the display or when no data is displayed even when the On/Off button is pressed.

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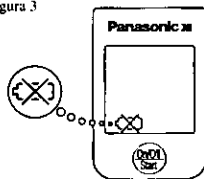CAUTION:

PRECAUCIÓN:

Do not throw batteries into a fire. Doing so may result in an accident or explosion.

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Positioning the Pressure Cuff

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CAUTIONS:

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- Do not use the unit for any purpose other than measuring blood pressure. Also, do not use the unit in conjunction with other devices. Improper use may cause a malfunction or personal injury.

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- Keep the unit out of the reach of children.

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1. Remove your watch and any jewelry from your left hand.

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2. Place the monitor on the palm side of your wrist, roughly 1.5 cm (1/2") below your palm as shown in Fig. 4, and secure it with the Velcro fastener so that it fits comfortably (is neither too tight nor too loose).

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
- The orange markings on the strap indicate a wrist circumference of 13.5 to 19.5 cm (5-1/4" to 7-3/4"). If these markings do not overlap, the circumference of your wrist is too large.

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3. Make sure that the display is on the inside of your wrist (palm side up). Also, center the unit on your wrist.

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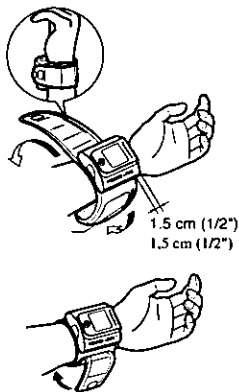When measuring your blood pressure at your wrist, the pressure cuff must be at heart level to ensure consistent, accurate readings.

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Taking measurements while sitting down

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1. Place your elbow on a table.

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2. Using the hard carrying case or other object as a support, position your wrist at heart level.

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- Your blood pressure measurement will be too low if you hold your hand too high, or too high if you hold your hand too low.

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3. Open and relax your hand with the palm facing up. Closing your hand will cause inaccurate readings.

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- Do not bend your wrist inward. Bending your wrist will cause inaccurate readings.

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4. Empty bladder, if possible, before taking blood pressure.

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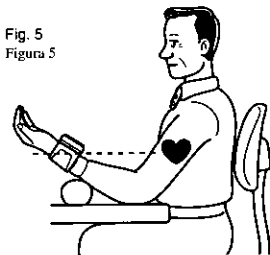
Fig. 5
Figura 5

Taking measurements while lying down

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1. Lie on your back.

Túmbese sobre su espalda.

2. Rest your wrist on a support or cushion with the palm side facing up; position your wrist at heart level.

Fig. 6
Figura 6

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3. **Open and relax your hand with the palm facing up. Closing your hand will cause inaccurate readings.**

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- **Do not bend your wrist inward. Bending your wrist will cause inaccurate readings.**

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4. **Empty bladder, if possible, before taking blood pressure.**

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IMPORTANT:

IMPORTANTE:

- **Measure your blood pressure at the same time and under the same conditions every day if possible. The ideal time to measure your blood pressure is in the morning immediately after waking up, while you are still relatively calm. If this is not possible, try to take your blood pressure prior to breakfast, before you become active.**

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- **Relax and take measurements in a comfortable position.**

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- **Rest for at least 5 minutes before taking a measurement.**

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- **Keep still and do not speak during measurement. Blood pressure can fluctuate with the slightest change, such as tensing your muscles or changing your posture.**

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Relax during measurements.

Relájese durante la medida.

1. To begin measurement, press the Start (On/Off) button (Fig. 7).

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- The LCD display will show 888 and then switch to 0. The pressure cuff in the strap will automatically inflate (Fig. 8).

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2. When finished, the blood pressure measurement is indicated on the display along with the pulse rate. The air in the pressure cuff is then automatically released (Fig. 9).

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Error

Error

- If "E" (meaning error) appears in the display (Fig.10) during measurement, an error has occurred, e.g., you have moved your hand during measurement. In this case, turn the unit off, wait at least 5 minutes and measure again.

Fig. 7
Figura 7

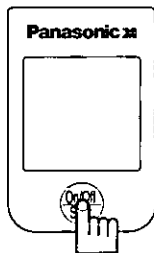
Fig. 8
Figura 8

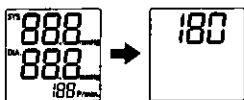
Fig. 9
Figura 9

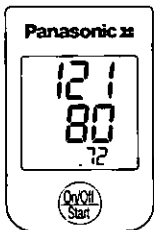
Fig. 10
Figura 10

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3. To switch the unit off press the Start (On/Off) button.

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- The unit switches off automatically after approximately 5 minutes.

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Note:

Nota:

- During the measurement, the unit may re-inflate once or twice in order to start at the appropriate pressure level. This is normal and is not an indication of a malfunction.

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CAUTION:

PRECAUCIÓN:

- For persons with poor circulation resulting from diabetes liver disease, arteriosclerosis, high blood pressure or other conditions, there may be significant differences in blood pressure values measured at the wrist versus at the upper arm.

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- Do not confuse self-monitoring with self-diagnosis.
No confunda autocomprobación con autodiagnóstico.
- NEVER change a prescribed medication without first consulting with your physician.

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Easily Check Your Blood Pressure Readings Against the World Health Organization (WHO) Guidelines

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IMPORTANT:**IMPORTANTE:**

- WHO does not define a low blood pressure area. For reference purposes, systolic pressure readings of less than 90 mmHg are considered to indicate low blood pressure regardless of the diastolic pressure.

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- Do not be alarmed by temporarily high or low readings because fluctuations in a person's blood pressure are not uncommon. If possible, measure and record your blood pressure at the same time every day, and consult your physician if you have questions or concerns.

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- If abnormal variations in blood pressure are observed in measurement, please consult your physician.

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About the Precise Logic™ Measurement System

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The blood flow (pulse wave) which influences blood pressure monitors varies according to the conditions of the blood vessels and the structure of the arm, making measurements by conventional methods difficult in some cases. For more consistent, accurate results, this device employs a Precise Logic™ measurement system that verifies the strength and shape of the pulse wave in order to reliably measure blood pressure.

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
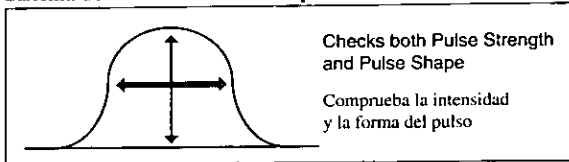
Old Technology

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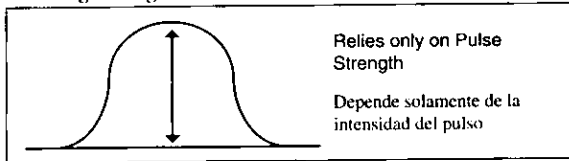By using the Precise Logic™ measurement system, measurement reliability and accuracy have been improved.

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Precautions to Ensure Safe, Reliable Operation

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Care and Maintenance

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1. Use a dry, soft cloth to clean the device. If desired, use a cloth lightly dampened with tap water and a mild detergent to clean the device. DO NOT use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.

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2. When the unit will not be used for a long time (30 days or more), be sure to remove the batteries. Otherwise, the batteries may leak and damage the unit.

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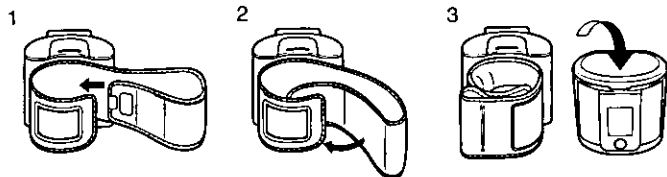
Storing the monitor

Para guardar el monitor

Fold the pressure cuff as shown in Fig. 11 and place the monitor into the hard carrying case.

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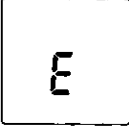
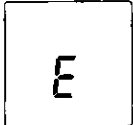
The measured value differs from that measured by the doctor.

The measured value is different for each measurement.

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- Blood pressure varies in response to minute changes in your mental state, such as your reaction to your doctor taking the measurement. One way to check this is to bring the unit to your doctor's office and directly compare the readings taken by your physician.

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- Relax for at least 5 minutes and then take another measurement.

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The measured value obtained is different from that measured at the upper arm.

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- For persons with certain circulation problems, there may be substantial differences in blood pressure values measured at the upper arm versus at the wrist.

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If the device still appears to provide unusual or erroneous readings, consult your physician. If the device does not appear to be functioning properly, contact Panasonic at 1-800-338-0552.

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Specifications are subject to change without notice.

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For questions or assistance with your blood pressure monitor, call us at 1-800-338-0552.

**Panasonic Personal and Professional
Products Company
A Unit of Matsushita Electric Corporation
of America**

One Panasonic Way 4A-3
Secaucus, NJ 07094

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>