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www.philips.com/AVENT

BPA-Free

BPA is Bispheno-A. This product uses Polyether Sulfone (PES), which is BPA-free and has a natural honey-colored tint.

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USA: Philips Consumer Lifestyle, a division of Philips Electronics North America Corporation.
1600 Summer Street, PO Box 120015,
Stamford, CT 06905-5125

Canada: Philips Electronics Ltd, 281 Hillmount Road,
Markham, ON L6C2S3

Trademarks owned by the Philips Group.
© 2008 Koninklijke Philips Electronics N.V. All Rights Reserved.
Las marcas son propiedad del Philips Group.
© 2008 Koninklijke Philips Electronics N.V. Todos los Derechos Reservados.
Les marques de commerce sont la propriété du Groupe Philips.
© 2008 Koninklijke Philips Electronics N.V. Tous droits réservés.

PHILIPS
AVENT

BPA-Free

Sin BPA

Sans BPA

EN On the Go Set

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FR Coffret Vie Active

42133 5411 020

EN GETTING STARTED

Please take time to read the Instruction Booklet to familiarize yourself with your pump. It is important that you know exactly how your Philips AVENT Manual Breast Pump works. You should use the pump at home until you feel completely at ease, before beginning to use it outside the home.

The On the Go Set has been specially designed for you to discreetly express and store your breast milk while away from your baby, then bring it home safely. You can maintain your milk supply by expressing at a time you would normally breastfeed and continue to breastfeed when you and your baby are together.

Before using your On the Go Set please consult your health professional or breastfeeding advisor regarding your breastfeeding and breast milk expression schedule.

Contents:

1x MANUAL BREAST PUMP

The Let-down Massage Cushion is designed to naturally imitate your baby suckling at the breast. This is intended to stimulate let down.

1x MICROFIBER INSULATED BAG

Specially designed by Philips AVENT, this stylish and discreet bag will carry your Manual Breast Pump and accessories and will keep your breast milk cool for several hours with the help of the cool packs provided.

2x BREAST MILK CONTAINER (2x 4oz/125ml)

Freezer and dishwasher safe. Express directly into these storage containers, which can also be used to feed milk. Just replace the sealing discs with AVENT Nipples.

2x FLEXIBLE COOL PACK

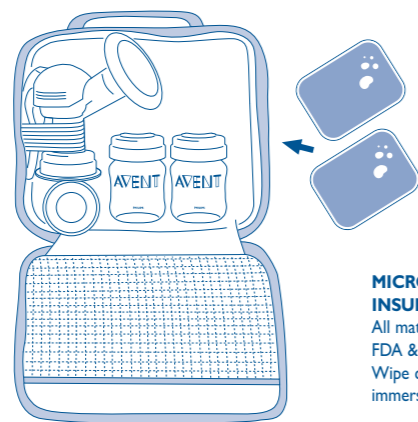
The cool packs must be placed in the freezer for at least 2 hours before each use. They are non-toxic and approved for use as described in this instruction leaflet. They remain flexible after freezing and can be refrozen again and again. To clean, wash with warm soapy water.

Caution: Do not swallow contents. Do not clean in a dishwasher. Suitable for use between 5°F and +158°F.

HOW TO PACK

Before you go out, make sure your breast pump body, breast milk containers and sealing discs are cleaned and sanitized. The pump body and breast milk storage components are sterilizer and dishwasher safe*. Always ensure your hands are clean before packing your breast pump into the bag, using the funnel cover and sealing base to keep the pump sterile and compact for travel, as shown.

Do not attach containers to the pump. Fit the sealing discs into the screw rings and screw them onto the milk containers. Pack the milk containers in the insulated travel bag with the two frozen cool packs taken from the freezer.

MICROFIBER INSULATED BAG
All materials conform with FDA & UK SI:3145 regulations.
Wipe clean only. Do not immerse in water. Made in China.

ON ARRIVAL

As soon as you arrive at your destination, put the cool packs in the freezer until you are ready to go home. The cool packs must be in the freezer 2 hours before use.

When you are ready to express, you must wash your hands thoroughly and clean your breasts and nipples with a clean damp cloth.

Remove the funnel cover and sealing base and screw a breast milk container onto the pump as shown in the Instruction Booklet. When you have finished pumping, unscrew the container from the pump body and seal with the screw ring fitted with its sealing disc. Place the container of expressed breast milk into the refrigerator immediately. You can use the drawstring bag provided for discreet storage.

Wash the pump body, replace the funnel cover and sealing base and place in the insulated bag. If you wish to use the pump again, you must re-sanitize.

When you are ready to leave, take the container of breast milk from the refrigerator. Take the cool packs from the freezer and place them inside the bag on top of the container. This will keep the milk cool during your journey home. Ensure containers are transported upright within the bag to prevent leaking.

Please Note: When removed from the freezer, two cool packs will keep either 1x 4oz/125ml or 1x 9oz/260ml breast milk container sufficiently chilled for several hours.

In hot climates, avoid storing the bag containing expressed breast milk in a car, office, cabinet etc. Milk must be sufficiently chilled to avoid spoilage.

Please note: Milk storage times vary based on collection and storage conditions. This storage information is only a suggestion. Please contact your health professional or breastfeeding advisor about milk storage guidelines.

WHEN YOU RETURN HOME

As soon as you reach home, put the cool packs into the freezer and the expressed breast milk into the refrigerator or freezer. Milk can be stored up to 48 hours (from first expression) in the refrigerator, 3 months in the freezer, or 6 months in a zero-degree freezer. Always store breast milk in the back of the refrigerator, not in the door. You MUST sanitize your Manual Breast Pump and breast milk containers before each use*. Clean the insulated bag with a soft damp cloth after each use.

* See the Instruction Booklet for full details.

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