
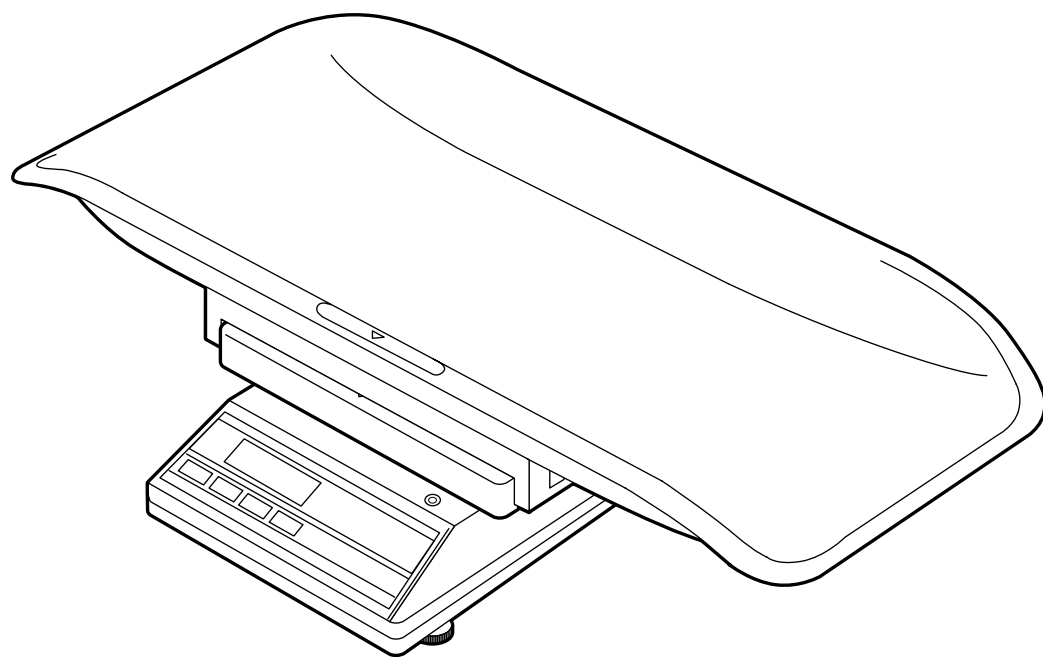
# TANITA

## ELECTRONIC SCALE FOR PAEDIATRIC / NEONATAL USE TL-150MA III INSTRUCTION MANUAL

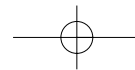Please read this Instruction Manual carefully and keep for future reference.

# Table of Contents

- Table of Contents .....3
- Applications .....4
- Safety Notes .....4
  - Caution Symbols .....4
  - ▲ Warning .....4
  - ▲ Caution .....4
  - ▲ [Maintenance] .....5
  - ▲ [Instructions for Measurement] .....5
  - <Usage Conditions> .....5
  - <Storage Conditions> .....5
  - <Power Source> .....5
- 1. Product Assembly and Components .....6**
  - Product Components .....6
  - How to Assemble the Equipment .....6
  - Control Panel .....7
  - Symbols and their Meanings .....7
- 2. Preparation for Use .....8**
  - Connection and Installation of the Equipment .....8
    - 1.Prepare Power Source .....8
    - 2.Level Adjustment .....8
- 3. Operating Instructions .....9**
  - Normal Measurement .....9
  - Tare Weighing Function .....10
  - Automatic Power off .....10
- 4. Troubleshooting .....11**
  - Troubleshooting .....11
- Specifications .....12

# Applications

1. This equipment has been calibrated as a precision instrument and can be used to certify babies' weights and/or to certify business transactions.
2. It can also be used to obtain reference data during medical examinations, such as periodic checkups, and to supervise babies' growth.

# Safety Notes

## Caution Symbols

In this Instruction Manual and the descriptions on the equipment, there are various caution symbols that will enable you to use this equipment safely and correctly so that harm to you, other people and loss of your property will be avoided.

The symbols and their meanings are as follows. Ensure you understand them well before proceeding with the Instruction Manual.

	<b>Warning</b>	This symbol indicates the risk of death or serious injury if the safety instructions are ignored and the equipment is mishandled.
	<b>Caution</b>	This symbol indicates the risk of physical injury and/or damage to the equipment if the safety instructions are ignored and the equipment is misused.
		This symbol indicates general precautions that should be taken when using this equipment.

## Warning

- To prevent electric shocks,  
Do not insert or remove the power plug with wet hands.
- To prevent fire hazards,  
Always use a correctly wired 230VAC, not multiple extension cords.
- To prevent the equipment toppling over, please make sure to fix the parts securely and accurately together. (See page 6 "How to Assemble the Equipment")

## Caution

- To prevent the spread of infections,  
Always clean the Weighing platform after use.  
Sterilize the platform with an appropriate alcohol-based cleaning fluid before each use.
- Installation of the equipment  
Ensure the weighing platform is placed on a level and stable surface. If the equipment is used when unstable, for example if not all feet are on a level surface, then there is a risk that scale will tip over and it will be impossible to obtain an accurate measurement.
- Measurement  
Always take the utmost care when placing a baby in the Baby Basket. Ensure that there is no risk of the equipment tipping over.  
To the equipment toppling over, always ensure that the weighing is properly supervised.
- AC adapter  
Ensure that the adapter supplied (MODEL: A30930G) is always used with this equipment. Other adapters may cause malfunctions.
- Battery  
When changing batteries, ensure that the new batteries are inserted with the correct polarity ⊕ and ⊖. If the polarity is incorrect, then the batteries may leak and damage the equipment.

- How to handle the equipment when moving it  
When carrying the equipment, please do not lift it up by the Baby Basket  
Please move the equipment by holding the main body firmly.
- The circuit board incorporates a fuse (100mA, 125V) for electronic equipment. If devices other than the original AC adapter are used or a large electrical current is applied, the fuse will be blown for safety purposes. If the fuse is blown, the unit must be inspected and repaired. As the fuse forms a fixed part of the unit and cannot be easily disassembled by the customer, please consult your nearest Tanita sales office or agent when the fuse needs to be replaced.
- Do not attempt to put fingers or objects between the inner plate and the scale unit to avoid injuries and malfunctions.

## [Maintenance]

This is a precision manufactured and accurately calibrated product, please observe the following instructions.

- Never disassemble or adjust the equipment, as this may cause malfunctions. The equipment should only be inspected in accordance with the regulations in each country.
- Do not apply excessive force to the equipment when assembling or when changing the batteries.
- Do not wipe the equipment with corrosive chemicals (benzine, cleaner, etc.). Please use a neutral detergent to clean the equipment.
- Avoid subjecting the equipment to excessive shocks or vibrations.
- When not in use for a long time, unplug the AC Adapter from the wall socket.
- When not in use for a long time, remove the batteries before storing the equipment.
- When transporting the equipment, use the box in which it was originally provided.
- When disposing of this unit, please do so in accordance with the prevailing regulations in each country.
- If an unauthorised person attempts to disassemble or repair any of the part, the warranty will become invalid. When the unit malfunctions, please consult your nearest Tanita sales office or agent.

## [Instructions for Measurement]

- Measurement may not be possible on a surface that is strongly vibrating. In such cases transfer the equipment to a stable surface with little or no vibration.
- When the equipment has been transferred to another location with a temperature difference of more than 20 °C, leave at least two hours before using.
- Do not use transmitters, such as mobile phones, near the equipment when in use. This may affect readings.
- Ensure that a clean towel or cloth is set on the baby basket when measuring a baby.

### <Usage Conditions>

Temperature Range for Use	: 5°C / 35°C
Relative Humidity	: 30% / 80% (without condensation)

### <Storage Conditions>

Temperature Range of Environment	: -10°C / 50°C
Range of Relative Humidity	: 10% / 90% (without condensation)
To avoid malfunctions, do not store the equipment in direct sunlight, significant temperature changes, the risk of dampness, a large amount of dust, in the vicinity of fires or where there is a risk of receiving vibrations or shocks.	

### <Power Source>

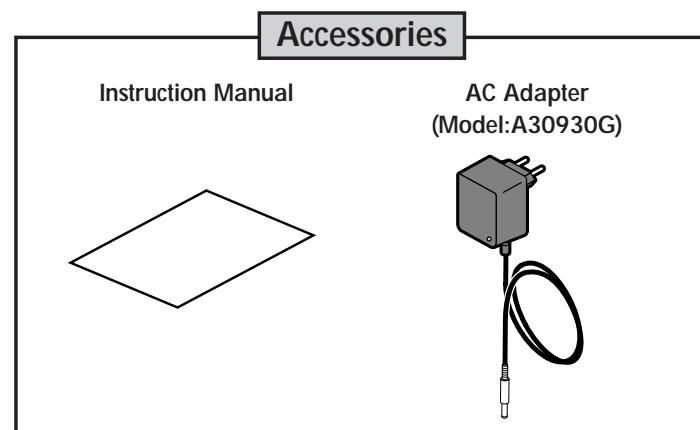
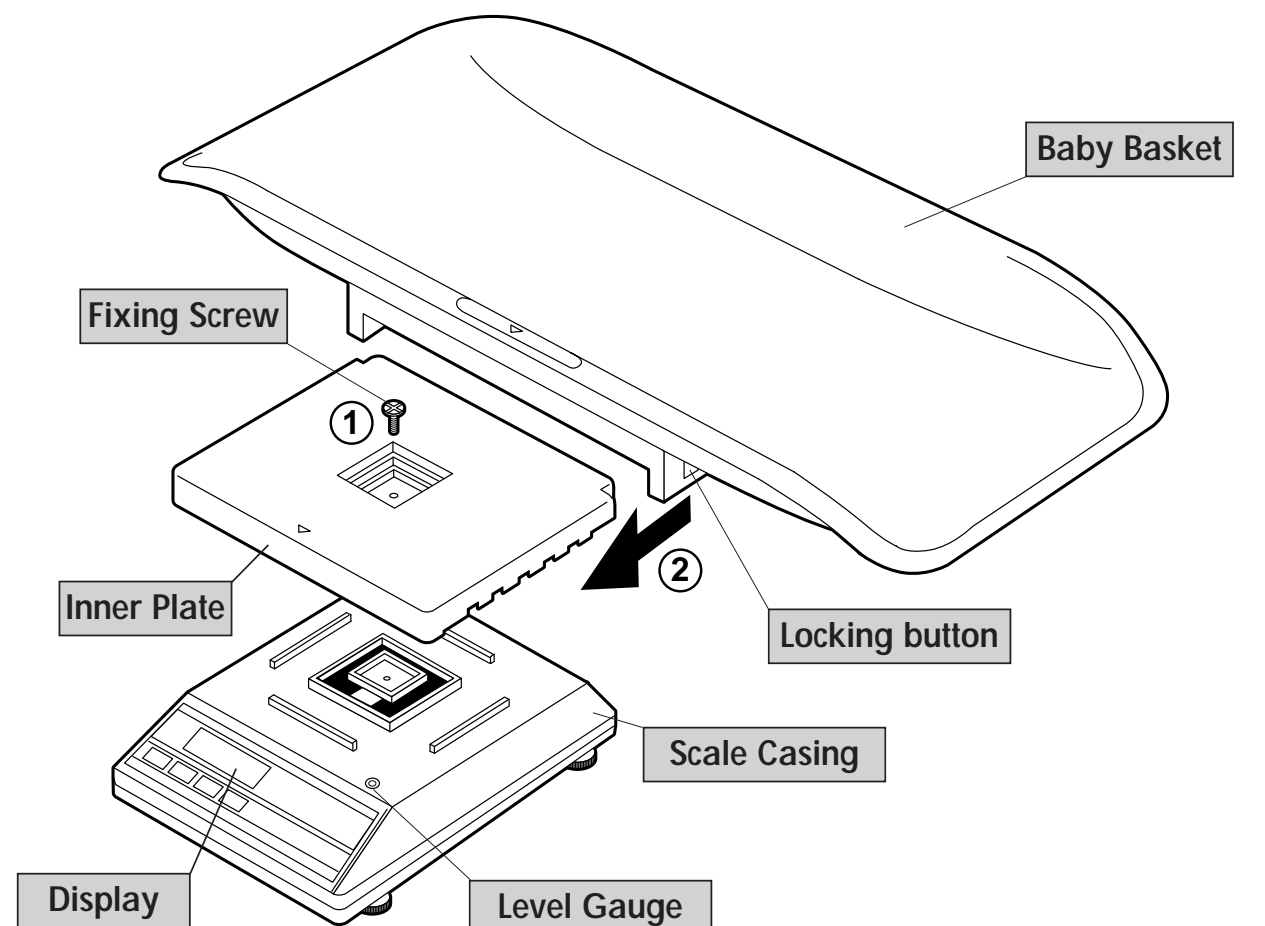
	Value Range
Voltage Range	230VAC
Frequency Range	50Hz
Electric Current Range	65mA

# 1.Product Assembly and Components

■ **Product Components** Ensure that the following components are included.

■ **How to Assemble the Equipment**

① Place the Inner Plate on the Scale Casing and fix it firmly with the Fixing Screws.

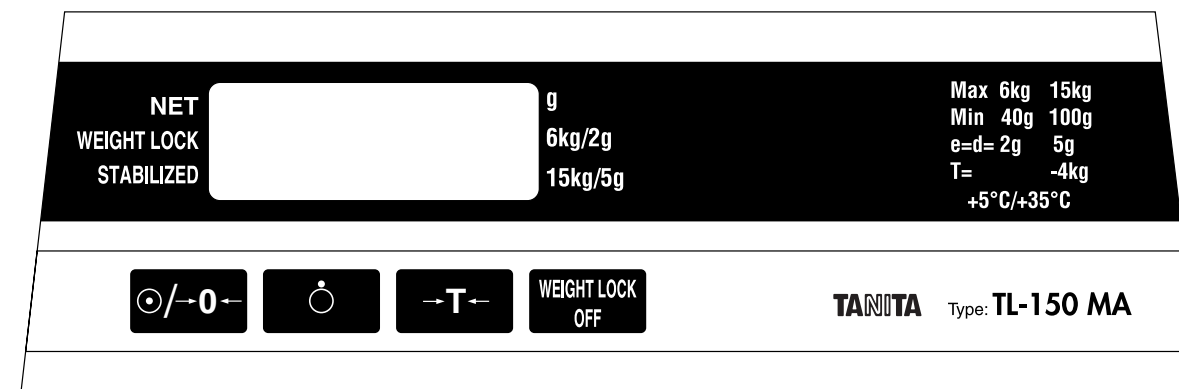
② Slide the Baby Basket from the rear of the Inner Plate to the front.

\* When removing the Baby Basket, please slide it to the rear while pushing the Locking button.

**Warning** Ensure that the parts are fixed securely and correctly to prevent the equipment from tipping over.

■ **Control Panel**

: Turn on the power of the Display and reset the Display to zero.

: Power off.

: Turn on the tare weighing function.

: "WEIGHT LOCK" function is deactivated.

NET : An arrow appears next to this icon when the tare weighing function is on.

WEIGHT LOCK : When the measurement becomes stable, an arrow will appear to the right of WEIGHT LOCK. When the "WEIGHT LOCK OFF" is pressed, the arrow will disappear.

STABILIZED : When the measurement becomes stable, an arrow will appear to the right of STABILIZED.

■ **Symbols and their Meanings**

	: The Display is switched on.		: Conformity with Medical Device Directive 93/42EEC		: Turns on the tare weighing function
	: Zero reset		: The Display is switched off.		: Type B applied part
	: Caution. Refer to this Instruction Manual		: Class II Equipment		: Direct Current

## 2. Preparation for Use

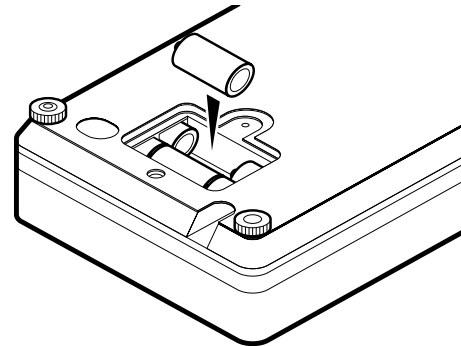
### ■ Connection and Installation of the Equipment

#### 1. Prepare Power Source

<When using dry cell batteries>

Please use six C alkaline manganese batteries (LR14) (not included).

Remove the lid of the Battery Compartment on the bottom of the Scale Unit Case, place batteries correctly as instructed inside the Battery Compartment and put the lid back on. If the batteries are running out, the Display will show **Lo**. In that case, please change all six batteries.

- ⚠ Do not apply excessive force to the equipment when assembling or when changing batteries.
- ⚠ When changing batteries, ensure that the new batteries are inserted with the correct polarity ⊕ and ⊖. If the polarity is incorrect, then the batteries may leak and damage the equipment.
- When not in use for a long time, remove the batteries before storing the equipment.

<When using the original AC adapter>


Connect the plug of the AC Adapter to the DC jack on the bottom of the Scale Casing and connect the AC Adapter to the wall outlet. Then confirm the Pilot Lamp on the AC Adapter lights up.

#### ⚠ Warning

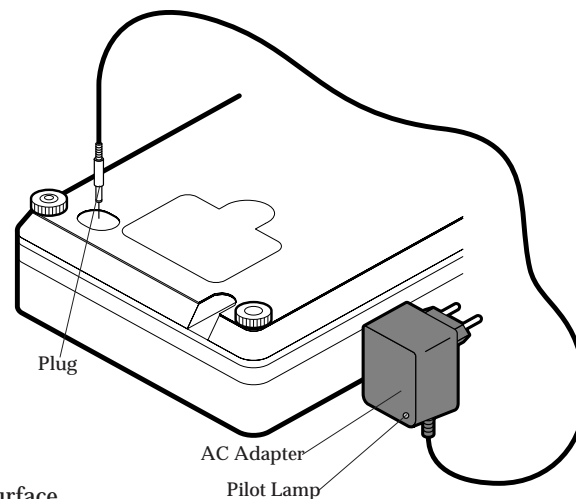- Do not insert or remove the plug with wet hands.
- Do not use the equipment near water.

#### ⚠ Caution

- Ensure that the adapter supplied (Model: A30930G) is always used with this equipment. Other adapters may cause malfunction.


#### 2. Level Adjustment

For accurate measurement, place the unit on as flat a surface as possible and adjust the level with the four adjustable feet so the bubble in the level gauge is in the center of the frame.

#### ⚠ Caution

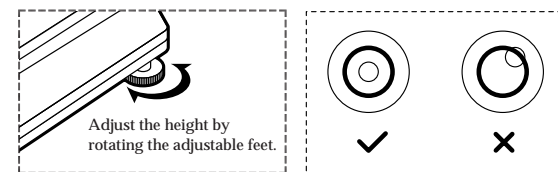
- Ensure the weighing platform is placed on a level and stable surface. If the equipment is used when unstable, for example if not all feet are on a level surface, then there is a risk that the scale will tip over and it will be impossible to obtain a measurement.

## 3. Operating Instructions

### ■ Normal Measurement (When Measuring without the Tare Weighing Function)

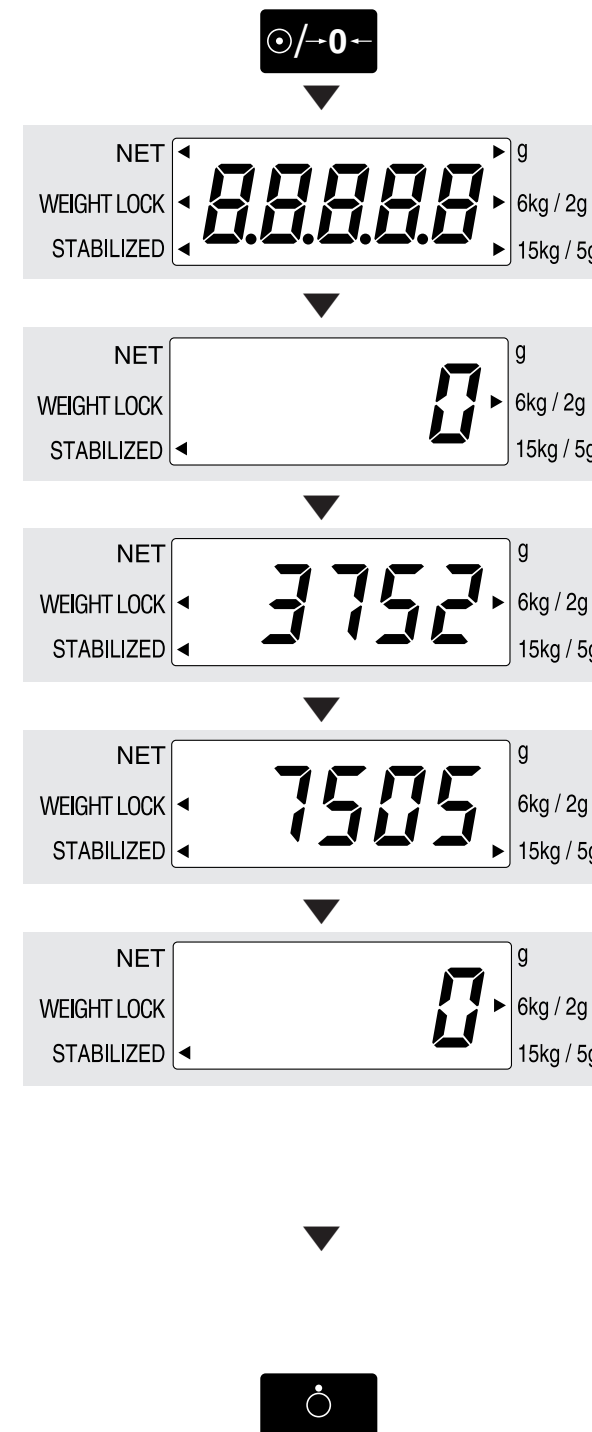- ⚠ When the equipment has been transferred to another location with a temperature difference of more than 20 °C, leave at least two hours before using.
- Do not use transmitters, such as mobile phones, near the equipment when in use. This may affect readings.
- ⚠ The 6kg mode has a resolution of 2g and the 15kg mode has a resolution of 5g.
- If the weight exceeds 6kg, the mode will switch to the 15kg version automatically.

#### 1. Turn on the Power

Press the **[ON/OFF]** key.

**88888** will flash for four seconds on the Display.

**0** will be displayed and an arrow will appear next to the "STABILIZED" symbol.

#### 2. Put the Baby into the Baby Basket

An arrow (◀) will appear next to "STABILIZED" and the weight will be displayed. The weight will be automatically locked in when the weight of the baby is stabilized and an arrow (▶) will appear next to "WEIGHT LOCK".

If the weight exceeds 6kg, the mode will switch to the 15kg version automatically.  
\* "Weight Lock" function fluctuates and does not lock-in any numbers when the weight of the baby is less than 500g. To deactivate "WEIGHT LOCK" function, press "WEIGHT LOCK OFF" key during or after measurement. "WEIGHT LOCK" function will automatically reactivate for subsequent measurements.

#### 3. Measurement is Completed

Remove the baby from the Baby Basket. **0** will be displayed and ▶ will appear next to the "STABILIZED" symbol. The mode will set at the 6kg version automatically.

⚠ If **0** is not displayed, press the **[ON/OFF]** key.

#### 4. Subsequent Measurements

After confirming the **0** is displayed, place the baby into the Baby Basket.

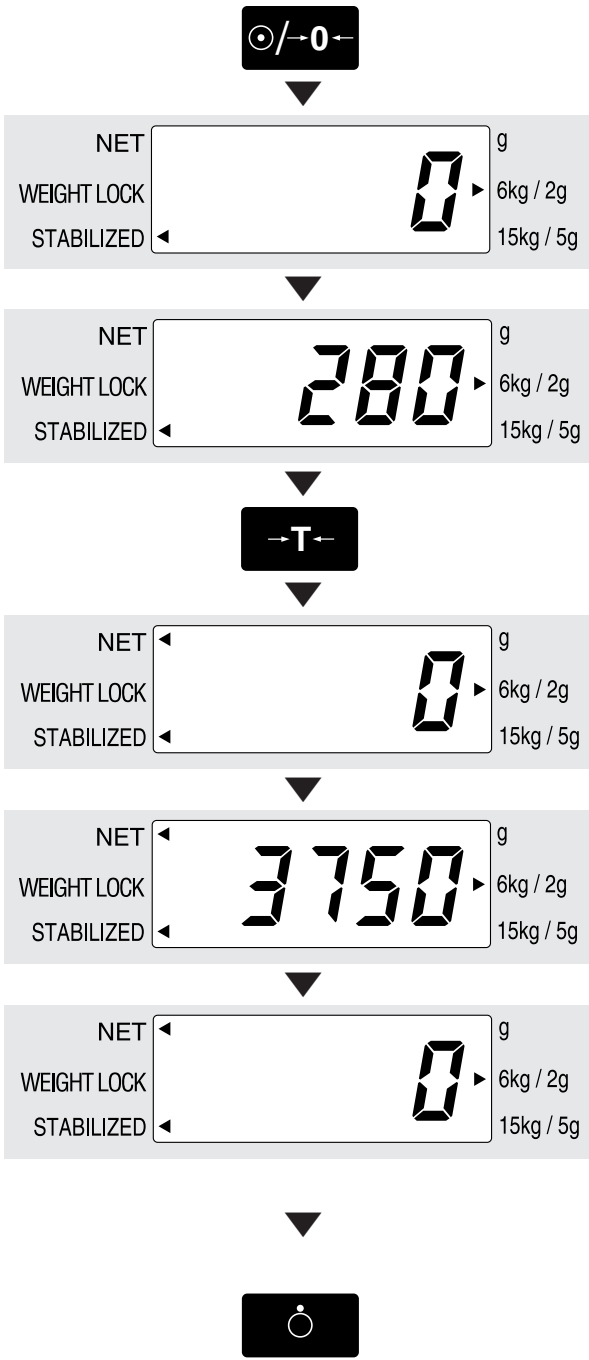
#### 5. Turn off the Power

Press the **[ON/OFF]** key.

# 4. Troubleshooting

## Tare Weighing Function (eg. Measurement if using Towels or Blankets)

- Tare weighing is possible only within the maximum tare weight. (4kg)
- When using the tare weighing function, the measurement range is reduced by the tare weight.  
 $[Measurement\ Range] = [Maximum\ Capacity] - [Tare\ Weight]$

- 1. Turn on the Power**  
Press the [ ] key.  
  
After flashes for four seconds on the display, will be displayed and ◀ will appear next to the "STABILIZED" symbol.
- 2. Place the baby's covering (towel, etc.) on the Baby Basket**  
An arrow (▶) will appear next to "STABILIZED" and the weight will be displayed.
- 3. Press the [ ] Key**  
 will be displayed and ◀ will appear next to "NET".
- 4. Place the Baby into the Baby Basket**  
An arrow (▶) will appear next to "STABILIZED" and the weight of the baby will be displayed.
- 5. Measurement is Completed**  
Remove the baby from the Baby Basket.  
 will be displayed and ◀ will appear next to "NET".
- 6. Subsequent Measurements**  
After confirming the is displayed, place the baby into the Baby Basket.
- 7. Turn off the Power**  
Press the [ ] key.

## Automatic Power off

This equipment has an automatic power off function that automatically turns off the power in order to reduce battery consumption.  
 If the equipment is not operated or the display does not change for ten minutes, the power will turn off automatically.

## Troubleshooting

If problems occur, please refer to the following instructions before you ask for repairs.

Problem	Solution
The unit does not turn on when the [  ] key is pressed.	<ul style="list-style-type: none"> <li>• When using the AC Adapter Ensure the AC Adapter cable is properly connected with the Display Unit. Ensure that the AC Adapter is correctly inserted in the mains socket.</li> <li>• When using cell batteries The batteries may have run out. confirm that the equipment is working by using the AC Adapter or new batteries.</li> </ul>
Measurement is not correct.	<ul style="list-style-type: none"> <li>• Check that all the adjustable feet are still resting on a flat surface after the level has been adjusted.</li> </ul>
Measurement does not stabilize. Display turns on and off.	<ul style="list-style-type: none"> <li>• Measurement may be impossible in the case of excessive vibration. Use the equipment in another location.</li> </ul>
is displayed.	<ul style="list-style-type: none"> <li>• It is possible that the maximum weight capacity has been exceeded.</li> <li>• Do not place a baby or objects into the Baby Basket before starting measurement. Start measurement after checking there is nothing in the Baby Basket.</li> </ul>

# Specifications

Model	TL-150MA
Power Source	AC adapter Centre minus MODEL A30930G CLASS 2 Input Voltage : 230VAC 50Hz 65mA Output Voltage : 9VDC±0.45VDC Rated Current : 300mA No Load Input Voltage : 12VDC±0.6VDC Alkali battery (LR14) x 6, (Not included)
Power Consumption	0.2W max.
Measurement System	Electrical resistance method
Maximum Capacity	6kg, 15kg
Minimum Graduation	6kg : 2g 15kg : 5g *Multiple range instrument
Accuracy at first calibration	6kg : ±4g 15kg : ±10g
Weighing Range	40g - 6kg 100g - 15kg
Maximum tare effect	4kg
Display	LCD (Liquid crystal) (Height of numerals 14mm)
Temperature Range of Usage	+5°C / +35°C
Relative Humidity	30 - 80% (without condensation)
Size	Baby basket
	Assembled Dimensions
Battery Life	approximately 300 hours of continuous use

The product meets all the requirements under the three standards below:

Safety Standard	: EN60601-1:1990, AM No.1:1993, AM No.2:1995 IEC60601-1:1988, AM No.1:1991, AM No.2:1995
EMC Standard	: EN-60601-1-2:1993 IEC60601-1-2:1993
Medical Device Directory Standard	: 93/42/EEC:1993

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### TANITA Corporation of America, inc.
Arlington Heights, Illinois 60005 U.S.A.
### TANITA Health Equipment H.K.LTD.
### TANITA Europe GmbH