



The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

CAUTION: TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SB300ss is warranted against manufacturing defects in materials and workmanship for one(1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SB300ss without charge for parts and labor. Simply send the SB300ss (postpaid) and a copy of your sales slip as proof of purchase to:

Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083



Made in China
Sonic Alert 1050 East Maple Rd. Troy, MI 48083
www.SonicAlert.com 1-800-566-3210 V/TTY



Sonic Alert

America's Leader in Visual Alerting Systems.

Sonic Boom Alarm Clock with Bed Shaker Model SB300ss




Warning: To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting, make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC 110V/220-340V (50Hz/60Hz)

Made in China
Sonic Alert 1050 East Maple Rd. Troy, MI 48083
www.SonicAlert.com 1-800-566-3210 V/TTY

LED ALARM CLOCK WITH VIBRATOR MODEL SB300SS

OPERATION INSTRUCTIONS

IMPORTANT- Please read these instructions carefully before use and retain for future reference.

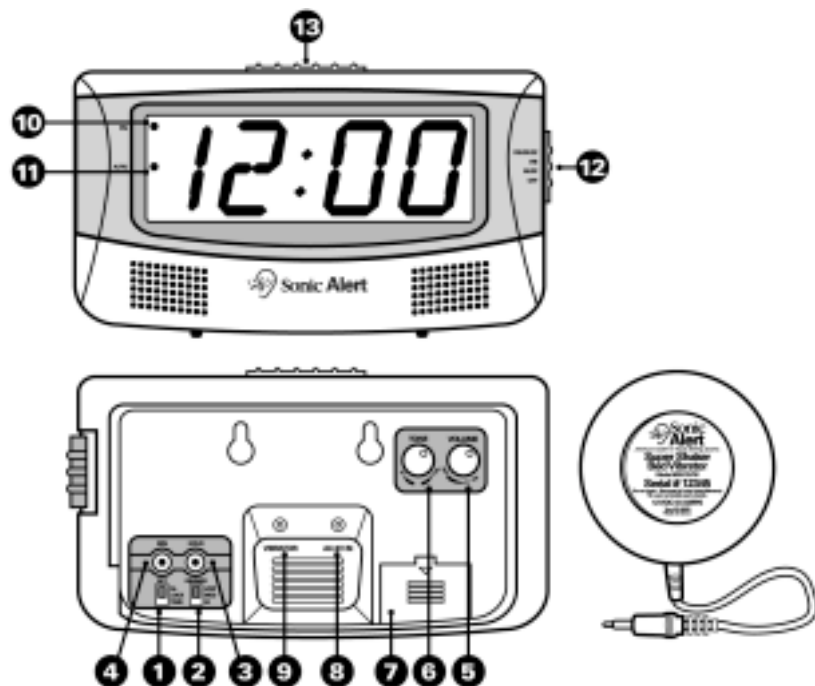

Features:

- | | |
|----------------------|-------------------------|
| 1) Alarm / Time set | 8) AC Power Input |
| 2) Dimmer Switch | 9) Vibrator Input |
| 3) Hour Set Button | 10) PM Button Indicator |
| 4) Minute Set Button | 11) Alarm Indicator |
| 5) Volume Knob | 12) Function Switch |
| 6) Tone Knob | 13) Snooze Button |
| 7) Battery Cover | |

Setting the correct time-

- 1) Slide the setting switch (1) to the position marked "TIME".
- 2) Depress the hour button as many times as necessary to advance the clock display until the hour is correct.
- 3) Depress the minute button as many times as necessary to advance the clock display until the minute is correct.
- 4) Slide the setting switch (1) back to the position marked "LOCK" This will prevent the clock time from being changed.

- 5) A lighted dot (10) in the upper left hand clock display indicates time is PM, no light indicates time is AM.

Setting the alarm time-

- 1) Simply follow the instructions mentioned under "Setting the correct time" except that you will have to slide switch (1) to the "AL" position.
- 2) Make sure the alarm time is correctly set for AM or PM lighted dot (10) indicates PM). Be sure to release the HOUR and MIN set buttons before releasing the ALARM button.

Snooze Operation-

SNOOZE button (13) press after alarm has sounded. The alarm will be turned off for 9 minutes for extra sleep. The alarm will come back again after 9 minutes. You may repeat this cycle many times, up to 59 minutes maximum.

Alarm-

Set the alarm time you wish and slide the VIB / BUZZ, VIB, BUZZ, OFF (8) switch to the desired position. VIB / BUZZ for both vibrator & sound, or VIB for vibrator only or BUZZ for sound only. To turn alarm off select the OFF position

Tone-

Adjust the tone (6) by rotating the tone wheel located on the back top right side.

Volume-

To adjust the volume (5) rotate the volume wheel located on back right side.

Bed Shaker / Vibrator- Figure B

- 1) Attach the bed shaker jack to the back of clock at VIBRATOR input.
- 2) To turn on vibrator slide switch (12) to VIB or VIB/BUZZ position.
- 3) Place vibrator under mattress or pillow.

AC-9V Power Input -

Attach the AC power supply input to the back of clock at AC-9V IN next to the VIBRATOR input, smaller input of the two. Important do not confuse input with vibrator input

Battery Backup-

To use battery backup, you can install a 9-volt alkaline battery in the back of the clock. Located under the battery cover (7). Battery not included.

GENERAL DATA

Power: 110 Volts, Consumption 8 watts, 60Hz
UL listed transformer (Underwriters Laboratories)

Made in China
Sonic Alert 1050 East Maple Rd. Troy, MI 48083
www.SonicAlert.com 1-800-566-3210 V/TTY

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>