

Sonic Siren Adjustable Remote Horn Model RH 100

OPERATION


The Sonic Siren Adjustable Remote Horn model # RH100, will alert when a signal is transmitted from a Sonic Alert Transmitter (Telephone Ring, Baby Cry and all Doorbell Signalers).

INSTALLATION

1. Plug the RH100 Remote Horn into any standard electrical outlet.
2. Select the volume level that you prefer with the adjustment wheel on the left side of the unit.
3. Select the tone level that works best with your hearing with the adjustment wheel on the right side of the unit.

Sonic Siren Adjustable Remote Horn RH 100

Diagram

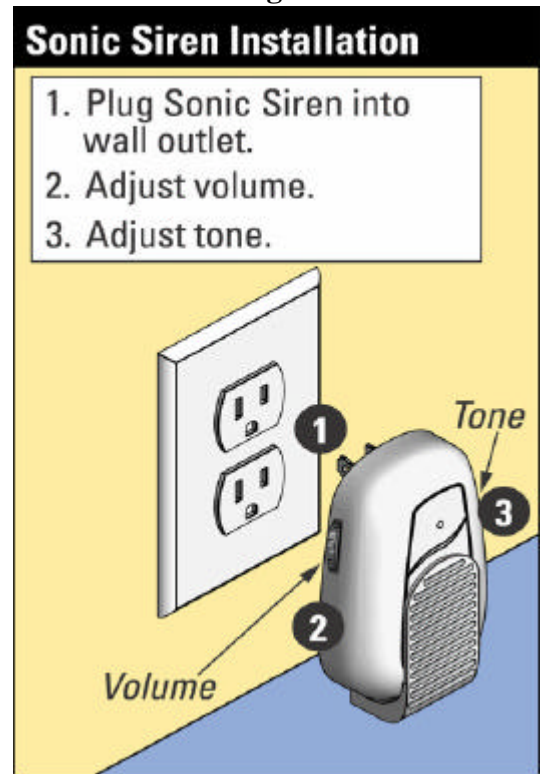

Sonic Alert's Limited Warranty

The Adjustable Remote Horn, RH100 is warranted against manufacturing defects in materials and workmanship for 5 years from the date of purchase. Within this period Sonic Alert will repair or replace at our option the RH100 without charge for parts or labor. Simply send your unit (postpaid) and a copy of your sales proof of purchase to: Sonic Alert, Inc. 1050 East Maple Rd., Troy MI 48083.

For Technical and Customer Service

Sonic Alert, Inc.
248-577-5400 V/TTY
e-mail: sonic-info@sonicalert.com
www.SonicAlert.com

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>