02134A

Please read the following instructions and warnings carefully.


Requires one 9V battery (not included).

Keep this instruction manual for future reference.

For Technical Support

E-Mail at customerservice@summerinfant.com

For prenatal use.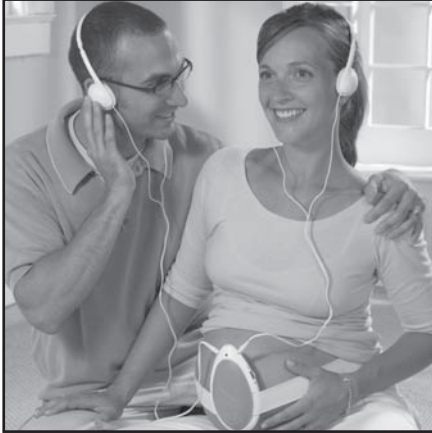

⚠ WARNING:
BELTS AND CABLES POSE A STRANGULATION DANGER TO YOUNG CHILDREN. WHEN NOT IN USE, STORE ENTIRE SYSTEM OUT OF REACH OF CHILDREN.

INSTRUCTION MANUAL

HEART-TO-HEART PRENATAL LISTENING SYSTEM™


The Heart-to-Heart Prenatal Listening System™ is easy to use. You will get the best results by reading the instructions before use. Included with your kit is a CD that demonstrates the actual sounds you will hear with the Heart-to-Heart Prenatal Listening System™ and provides basic instructions for use. The Heart-to-Heart Prenatal Listening System™ is a wonderful way to hear and share the precious sounds of your baby's tiny heart. It is a completely safe and non-invasive way to listen to your unborn baby's heartbeats. Unlike ultrasound, it emits no energy into you or your baby.

NOTE: The Heart-to-Heart Prenatal Listening System™ is NOT a medical device. It should NOT be used or relied upon for diagnostic or other medical purposes. This product should not be used in place of regular prenatal care by your physician.


The Heart-to-Heart Prenatal Listening System™ can begin to detect your baby's heartbeats during the second half of your second trimester — typically between 21-28 weeks. See page 5 for factors influencing the ability to hear your baby's heartbeat.

FEATURES & COMPONENTS:


SENSOR UNIT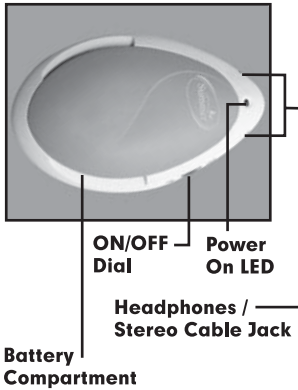


HEADPHONES (2)

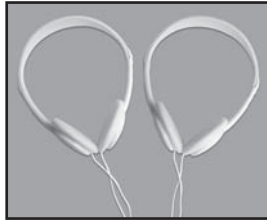
POSITIONING BELT

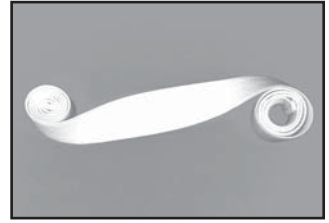RECORDING CABLE

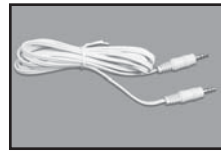INFORMATIONAL CD

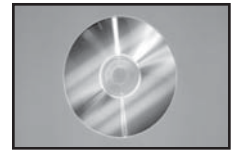SET UP & USE:


Installing the Battery and Belt

Step 1: Remove the sensor cover. Place your thumb and forefinger in the belt channel indentations at the wide end of the cover (Figure A). Pull the cover up and then slide it out away from the narrow end (Figure B).

Step 2: Install a 9 volt battery (not included). Place the battery into the rectangular battery compartment (Figure C). Be sure to match the + and - symbols on the battery with those on the compartment. Slide base in first and then snap in the contact end (Figure D).


Note: To change the battery, lift up on the contact end and pull the battery out. The sensor cover may be used to pry the battery up (Figure E). For more on battery, see 'Battery' on page 6.

Step 3: Place the belt in the belt channel (Figure F).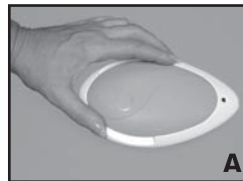

SET UP & USE (continued):

Step 4: Replace the sensor cover. Hold the sensor cover at the wide end. Insert the narrow end first, and then snap the wide end closed (Figure G). The belt should slide freely through the belt channel when the cover is closed (Figure H).

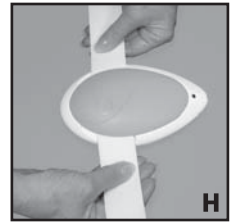Using your Heart-to-Heart Prenatal Listening System™:

Step 1: Loop the belt around your abdomen. Use the hook and loop tab to adjust the belt until it is snug, yet comfortable, with the Heart-to-Heart Prenatal Listening System™ sensor just below your navel.

The purpose of the belt is to hold the sensor in place and snug against your skin. You can use other methods to hold it in place. You may find it more comfortable to position the sensor underneath the snug-fitting mesh of maternity pants.

WARNING: Belt poses a strangulation danger to children. When not in use, store system with belt out of reach of children.

Step 2: Get into a reclining position. You can hear your baby best while lying or reclining in bed, on a sofa, or in a recliner with your upper body elevated by pillows. For comfort, elevate your right hip and/or knees with a small pillow.

WARNING: Avoid lying flat on your back for an extended period of time during pregnancy, especially in your last trimester. Check with your health care professional for recommendations on this matter.

Step 3: Turn power on using the volume switch on the side of the product (Figure I). The indicator light will illuminate when the power is on. If the light does not illuminate, the battery power is very low and the battery must be replaced. Install a new 9 volt battery and try again. (For information on battery life, see "Battery" page 6.)

Step 4: Plug up to two sets of headphones into the jacks on the sides of the unit to listen (Figure J).

SET UP & USE (continued):

Step 4: Position the sensor. Use the illustration on this page to help you position the sensor. You may need to try the sensor in several locations before you locate the rapid drumming of your baby's heart. Be sure to remove your hand after you reposition the sensor. When you move or touch the sensor with your hand, you'll notice an immediate drop in volume. That's because the Heart-to-Heart Prenatal Listening System™ sensor is designed to automatically soften the louder sounds that result when you touch or move it. The volume will return to normal when you remove your hand.

- Be sure the sensor is touching your bare skin and the belt is holding the sensor snugly in place.
- During the early stages of your pregnancy, your baby may be low in your abdomen. As your baby grows, it takes up more space and you may hear the heartbeat higher in your abdomen. The illustration on this page may help you find the best sensor location.

Step 5: Listen. Some of the sounds you hear may be difficult to interpret. Use the Sound Guide at the bottom of this page to identify the sounds you hear.


Third Trimester

Second Trimester

(To get the clearest reproduction of your baby's heartbeat, see "Factors Influencing the Ability to Hear Your Baby's Heartbeat" on pages 5, 6 & 7 of this manual.)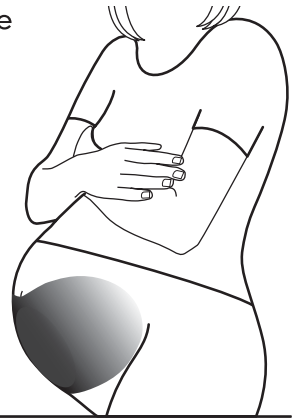

SOUND GUIDE:

Listen to the enclosed CD for the sample sounds.

Fairly steady, rapid drumming. This is the sound of your precious baby's heartbeat. It is not unusual for the pace of the heartbeat to vary while you are listening.

Strong thump. You've felt your baby kick and now you may be hearing it, too.

Multiple thumps. Your baby may have hiccups. Hiccups are typically 3 to 5 seconds apart. You may feel these thumps as your baby moves with each hiccup.

SOUND GUIDE (continued):

Growling. This is your stomach growling or your intestine gurgling. These noises are not associated with your baby.

Soft rumbling. If your baby is not in an optimum position for listening, you may hear a low rumbling sound. Try again later after your baby has shifted. If you are in the early stages of your pregnancy, you may also hear this sound. The Heart-to-Heart Prenatal Listening System™ usually can begin to detect your baby's heartbeats during the second half of your second trimester — typically between 21 and 28 weeks.

Your heartbeat. You can also use your Heart-to-Heart Prenatal Listening System™ to listen to your own or your child's heartbeat. In this case, you can hold the sensor to your chest or your child's chest with your hand.

FACTORS INFLUENCING THE ABILITY TO HEAR YOUR BABY'S HEARTBEAT:

Do not be alarmed if you do not hear your baby's heartbeat right away. The following factors affect when and how well you will hear your baby's heart beating.

Do Not Hold the Sensor

The positioning belt provided is designed to hold the sensor snugly in place in order to get the best sounds. After positioning the sensor, remove your hands and listen. Do NOT hold the sensor in position with your hand. Doing so will interfere with its ability to detect your baby's tiny heartbeat. Be sure to remove your hand after you reposition the sensor. When you move or touch the sensor with your hand, you'll notice an immediate drop in volume. That's because the Heart-to-Heart Prenatal Listening System™ sensor is designed to automatically soften the louder sounds that result when you touch or move it. The volume will return to normal when you remove your hand.

Stage of Your Pregnancy

The Heart-to-Heart Prenatal Listening System™ will usually begin to detect baby sounds during the second half of your second trimester (typically 21-28 weeks). At this stage of your pregnancy, adjust the position of the Heart-to-Heart Prenatal Listening System™ sensor so that it is between your navel and pubic bone. As your baby grows, you will hear its heartbeat with the sensor farther up your abdomen, closer to your navel. (See illustration on page 4.)

FACTORS INFLUENCING THE ABILITY TO HEAR YOUR BABY'S HEARTBEAT (continued):

Do not be alarmed if you do not hear your baby's heartbeat right away. The following factors affect when and how well you will hear your baby's heart beating.

Baby's Position and Activity

Your baby is quite active and moves often. It is not unusual for your baby to move into a position where the heartbeat cannot be heard, especially during the earlier months of your pregnancy. If your baby moves while you are listening, simply change the position of the sensor or try again later. (After repositioning, be sure to remove your hands from the sensor.) If your baby is particularly active, the kicking sounds may make it difficult to hear the drumming of the tiny heartbeats.

Mother's Weight

Your weight before and during pregnancy may affect how early the Heart-to-Heart Prenatal Listening System™ can detect your baby's heartbeat. A slender mother will generally hear the baby's heartbeat earlier in her pregnancy. The Heart-to-Heart Prenatal Listening System™ is a passive listening device and, therefore, it has no means of penetrating your body's insulating layer of fat. This means that moms with a larger percentage of body fat may have to wait a few more weeks until the baby's tiny heartbeats become strong enough to be heard.

Mother's Position

You can hear your baby while lying or reclining in bed, on a sofa, or in a recliner with your upper body elevated by pillows. For comfort, elevate your right hip and/or knees with a small pillow.

WARNING: Avoid lying flat on your back for an extended period of time during pregnancy, especially in your last trimester. Check with your health care professional for recommendations on this matter.

Battery

The Heart-to-Heart Prenatal Listening System™ operates on a standard 9-volt battery. If the light does not illuminate when the headphones are connected, try disconnecting and reconnecting the headphone jack. If the light still does not illuminate, your battery is very low or completely drained. A standard battery will last 10 to 15 hours under normal use. For best results we recommend the use of an alkaline battery. (Even when your battery is very low, when the headphones are plugged in your sensor is operating and will continue to detect sounds but much less effectively.)

NOTE: The Heart-to-Heart Prenatal Listening System™ is a fun, recreational device for listening to your baby's sounds. It is NOT a medical device. It should NOT be used or relied upon in any way for diagnostic or other medical purposes.

FACTORS INFLUENCING THE ABILITY TO HEAR YOUR BABY'S HEARTBEAT (continued):

Do not be alarmed if you do not hear your baby's heartbeat right away. The following factors affect when and how well you will hear your baby's heart beating.

Listening and recording

Hearing your baby's heartbeats and other sounds is a precious, exciting experience. You can listen to those sounds with your own headphones or those provided with the Heart-to-Heart Prenatal Listening System™. You can also use your personal computer to record and email these sounds to your family and friends.

To listen with headphones

Simply plug headphones into the headphone jack. The headphones supplied with your system are specifically tuned to fetal heart sounds. You can use almost any stereo headphones with the Heart-to-Heart Prenatal Listening System™; however, some headphones may not adequately reproduce the sounds.

To record your baby's sounds

Connect the recording cable between either jack on the side of the Heart-to-Heart Prenatal Listening System™ and the microphone (or line in) input on your personal computer. You are now ready to record a sound file (see your computer system user manual for recording. Microsoft Windows users can use the Sound Recorder program). For best results, set your personal computer's microphone (or line in) volume to low and adjust the main unit's volume until you achieve an acceptable recording sound. Once this file has been created and saved, you may create an email and attach the sound file to share with family and friends.

CARE & MAINTENANCE:

To clean the sensor, headphones, and cables, simply wipe them with a damp cloth.

To wash the belt, remove it from the sensor unit and fasten the hook and loop closure. Wash with similar colors in the gentle cycle of your washing machine; tumble dry using the delicate cycle of your dryer.

The Heart-to-Heart Prenatal Listening System™ is carefully designed to ensure optimum sensitivity. Like other sophisticated electronic equipment, The Heart-to-Heart Prenatal Listening System™ can be damaged if not handled with care. In addition, the belts and cables may pose a danger to young children. Therefore, please follow these safety precautions:

- Always disconnect headphones when not in use.
- Keep your Heart-to-Heart Prenatal Listening System™ out of water.
- When not in use store your Heart-to-Heart Prenatal Listening System™ out of reach of children.
- Do not drop your Heart-to-Heart Prenatal Listening System™.
- If you will be storing your system for an extended period of time, be sure to remove your battery. Battery corrosion that may occur will damage the unit.

FCC Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

www.summerinfant.com

We at Summer Infant stand behind all of our products. If you are not completely satisfied or have any questions, please contact our Customer Service Department at 1-800-268-6237 or e-mail us at customerservice@summerinfant.com.

Summer Infant, Inc.
582 Great Road
PO Box 829
Slatersville, RI 02876-0899 USA
1-800-268-6237

Summer Infant Europe, LTD.
20 Sparrows Herne
Bushey, Hertfordshire
WD23 1FX UK
44 (0)20 8420 4429

© 2005 Summer Infant, Inc.
Colors and styles may vary.
MADE IN CHINA.
Please retain information for
future reference.

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