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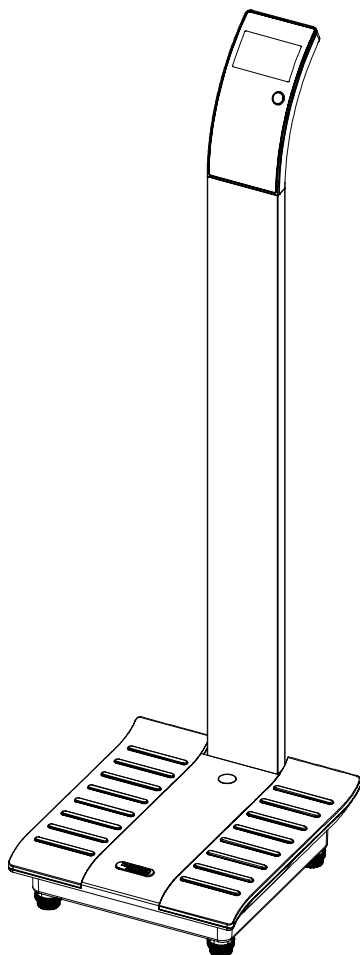
## ELECTRONIC MEDICAL SCALE CLASSE III

Dear Customer,

Thank you for purchasing this product. We hope you will find it satisfactory.

Please read these instructions carefully to take full advantage of your product.

## READ BEFORE USE

The Electronic Medical Scale is designed and manufactured in a facility certified in the following Management Systems: ISO 9001 Quality, ISO14001 Environment and OHSAS18001 Health and Safety.

### APPLICATIONS

The electronic medical scale's intended use is for weighing patients for the purposes of monitoring, diagnosis and medical treatment.

### PRECAUTIONS OF USE

		This symbol indicates general precautions that should be taken when using this product.
	<b>WARNING</b>	This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.
	<b>CAUTION</b>	This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

### WARNING

- Use the product on a firm and flat surface.
- Clean after use with a lightly dampened cloth without allowing any water to get into the device. Do not use solvents or immerse the product in water.
- Avoid excessive impact or vibration to the product, such as dropping the product onto the floor.
- Do not jump on the weighing platform.

### CAUTION

- Do not disassemble the scale as incorrect handling may cause injury.

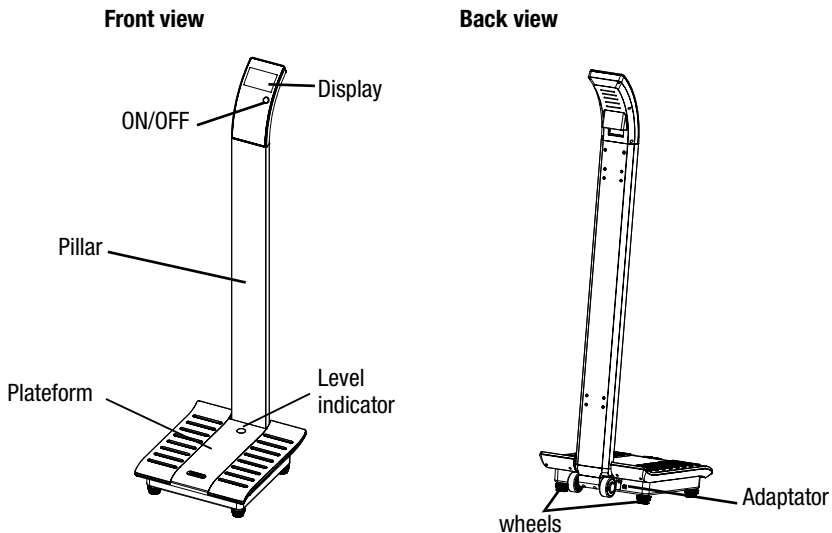
## MAINTENANCE

- The product has been manufactured with precision. The user should not disassemble the product on its own.
- Remove batteries if the product is not in use for an extended period.

## ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III

## PRODUCT SPECIFICATION

- Class I medical device (EC Directive 93/42 EEC).
- Class III OIML compliant.
- Class III Non-Automatic Weighing Instruments (EC Directive 90/384 EEC) compliant.
- Level indicator and adjustable feet.
- On/off function
- Capacity: 220 kg.
- Graduation: 100 g.
- Operates through 1.5 V X 6 (AA/ LR6) or with AC 9V [500 mA] adaptor (adaptor included).

## ITEMS INCLUDED IN THE PACKAGE

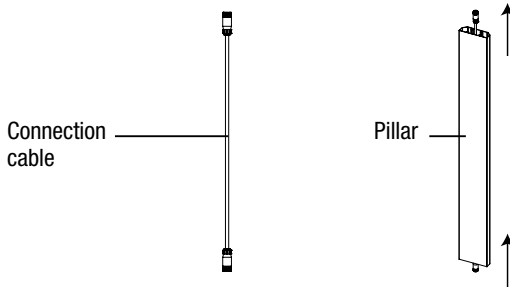
- 1 display
- 1 connection cable.
- 1 pillar.
- 1 Base part with platform and adjustable feet.
- 3 screws M5x15.
- 6 screws M4x45.
- 1 screw driver
- Instruction and assembly manual.
- Metrological book.

## ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III


## PROCEDURE

## SCREEN INSTALLATION AND CONNECTION


1. Insert the connection cable into the pillar

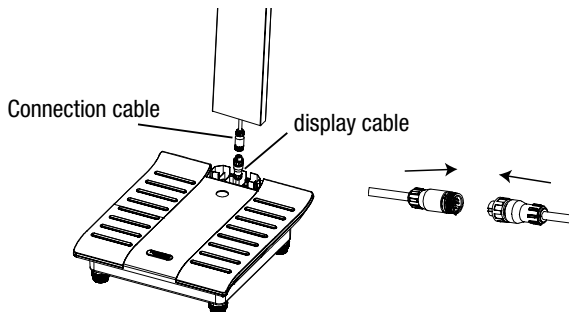2. Connect the platform connection cable with the display cable and screw together check the pillar mounting direction. An arrow on the pillar indicates which end to insert into the platform.

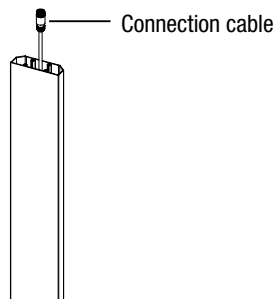
3. Deploy the cable and then insert the pillar in the space provided for this purpose on the platform

## ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III

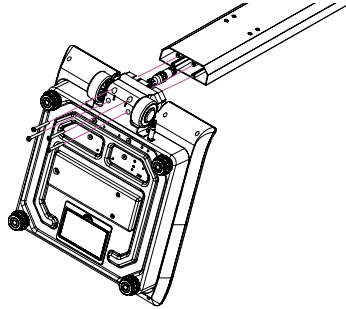
4. Using the screwdriver, attach the pillar to the platform by screwing 4 screws M4x45 slightly beneath the platform.

5. Connect the pillar cable to the screen and screw.

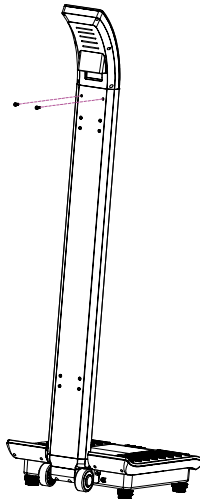6. Insert the screen into the pillar and attach with 2 M5x15 screws


7. Set the balance.

## ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III
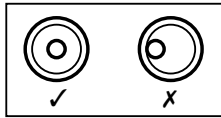
## SETTING THE BALANCE

• Before you use the scale for the first time or when the scale has been moved to another place, the balance of the scale must be set.


1. Do not step on the scale or/and remove all weight from the platform.
2. Place the scale on a hard and flat surface, use the adjustable feet to adjust the balance until the bubble inside the level indicator is centrally located in the circle.


## PRODUCT USAGE

## INITIALISATION

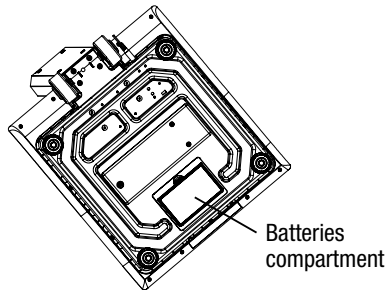1. Insert 6pc AA battery into the battery compartment located at the back of the base, or power up the scale through connecting the DC adaptor with the power socket.
2. Remove the batteries if the scale has not been used for an extended period.

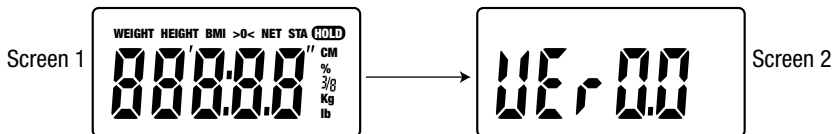3. When the scale is powered up, the software version will be displayed for a few seconds (see below), and then all.


The scales will switch off automatically after 2 seconds.

# ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III
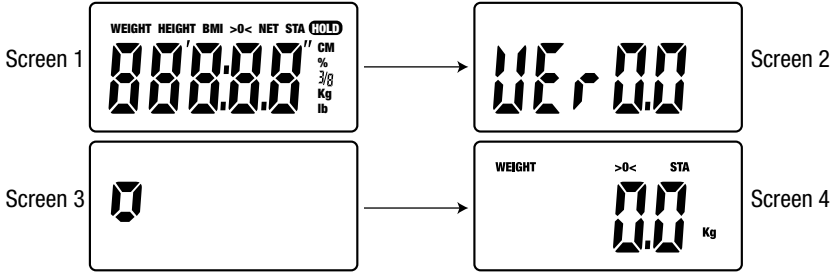
## OPERATION

Weighing only operation

1. Press the button (🔌/OFF) to power up the scale The number 0 scrolls on the screen, then the display settles on "0.0".

2. Wait until screen displays "0.0" before stepping on the scale. Then, stand still while your weight is being computed. The screen will display your weight automatically.

3. Step off the scale, the display returns to « 0.0kg ».
4. Press the (🔌/OFF) button to turn the scale off immediately or wait 10 sec, then it will turn off automatically.

## WARNING INDICATORS

Err0	Initial zero error. Step off the scale, wait until the scale shows 0.0 and repeat measurement.	AdErr	Machinery Error. Please contact your sales representative.
		EEP	Machinery Error. Please contact your sales representative.
bAtt	Low battery. Replace the batteries.	Ustb	Unstable zero. Step off the scale, then step on the scale again.

## ELECTRONIC MEDICAL SCALE MEDICALE CLASSE III

## PROTECT THE ENVIRONMENT

Dispose of used batteries in a designed container so that they can be collected and recycled. Do not mix different types of batteries. Do not mix new and used batteries.

After life, dispose of this product at a designated waste recycling point.

## GUARANTEE

This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be repaired free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.

## AFTER SALES SERVICE

For calibration and after sales services, please contact Terrailon consumer service at 0 826 88 1789 or at the following address: [serviceconsommateurs@terraillon.fr](mailto:serviceconsommateurs@terraillon.fr)

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### **Terrailon UK Ltd**
### **Terrailon Deutschland GmbH**
### **Terrailon Asia Pacific Ltd**
### **Terrailon Corp**