



Electronic Hearing Aid Dryer

Designed to remove moisture from your hearing aids and earphones and help to maintain peak performance.

Operating Instructions

In the Evening:

1. Push the switch on the back of the Dri-Dock to the ON position. The green light indicator should illuminate.
2. Place your hearing aid(s) inside the Dri-Dock with the battery compartment of the hearing aid left open. Place the earmold with the canal aimed down. This is the same for both in-the-ear and behind-the-ear hearing aids.
3. Close the lid of the Dri-Dock. The Dri-Dock will remove moisture from your hearing aid(s) while you sleep.

In the Morning:

1. Turn the switch on the back of the Dri-Dock to the OFF position. Open the lid and remove your hearing aid(s). Allow them to cool to room temperature.
 2. Wipe the hearing aid(s) with a dry tissue.
3. Insert hearing aid(s) and enjoy another day of better hearing.

Should you experience any difficulty with your hearing aids, contact your hearing healthcare professional immediately.

Limited Warranty:

The warranty for the TV Ears Dri-Dock Electronic Hearing Aid Dryer is limited to defects in material and workmanship for 1 year from the date of purchase. Warranty is voided should any repair be attempted by anyone other than TV Ears, Inc.

For service, contact TV Ears, Inc. toll free at 888-883-3277

Monday - Friday 7am-4pm PST

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>