




TV · EARS®

Dr. Recommended TV Listening Devices

TV Ears Link System

Creates a TV Ears looped environment for T-coil hearing aid wearers as they listen to their television. Now users can keep their T-coil hearing aids in and hear the clarity of dialogue on the TV. TV Ears Link is compatible with a TV Ears 2.3MHz system.

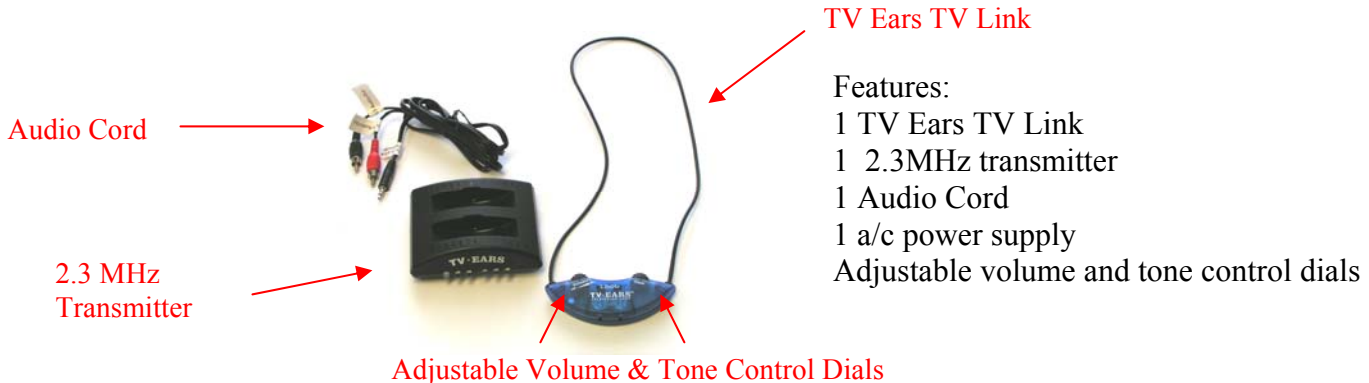

Step 1: Install Power

- Plug the AC power supply into the wall outlet and plug the other end into the 12v jack located on the back of the transmitter. Always keep transmitter plugged into a powered wall outlet.
- The 5 clear lights on the front of the transmitter are the LED infrared lights. They will glow red at all times when transmitter is receiving power. Cup you hand over the lights to confirm the transmitter has power.

Plug the AC Power Supply into the 12v jack on the back of the transmitter



The 5 LED infrared lights on the front of the transmitter



Step 2: Install Audio

- Insert the single end of the Audio Cord into the audio jack located on the back of the transmitter.

Audio port on back of your TV Ears transmitter



- Insert the double end of the audio cord audio out ports on the back of your television.

Example of Audio Out



Example of Audio Out



IF YOU HAVE NO AUDIO OUT PORTS ON YOUR TELEVISION:

You can use the audio out ports located on the back of your cable box or satellite box. If you do not have a cable or satellite box, you will need to purchase an Alternate Installation Kit. Please contact TV Ears **Customer Service toll free 1-888-883-3277 for assistance, or visit www.tvears.com.**

Step 3: Charge TV Ears 2Link

- Place the TV Ears Link in one of the cradles of the transmitter for 1 hour before use. The light on the left will be GREEN if placed in the front cradle or RED if placed in the back cradle. This will provide a full charge.


NOTE: the volume dial should be turned to the off position. You will hear/feel a click.

Far left light will indicate when charging



Step 4: Ready to Wear !!

- Turn your television on.
- Switch your Hearing Aid to the Amplification T- coil mode.
- Place the TV Ears Link over your head and wear like a necklace. It should rest on your chest upper chest.
- Turn the TV Ears Link on and adjust your volume level.
- You can also adjust tone for your hearing.

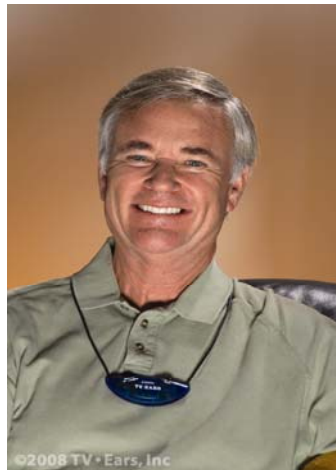

Happy TV Listening !!

Contact TV Ears Customer Service
Toll Free: 1-888-883-3277
Monday thru Friday, 7:00 a.m – 4:00 p.m. Pacific Time

Visit our Website: www.tvears.com

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>